



WARRANTY CARD

Customer Name: _____


Purchase Date: _____

Model No: HM110M Infant Height Measurement Mat

Distributor Stamp

Measurement Range: 10-110 cm (4-43 inch)

Graduation: 0.5 cm (1/4 inch)

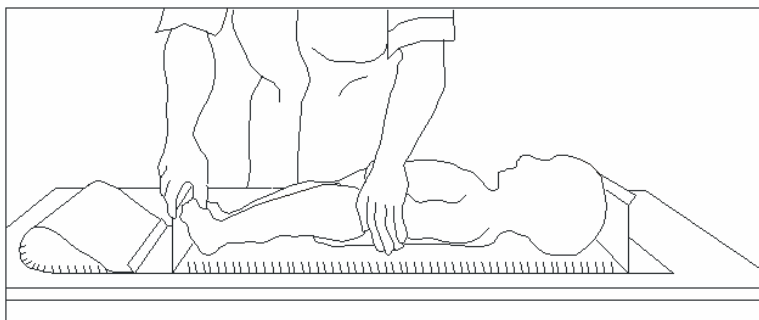

To conduct height measurement, lay subject on mat, with head touching head stopper. Fold over foot stopper and note height marking for results.

Thank you for choosing this Charder Medical device. It is designed to be easy and straightforward to operate, but if you encounter any problems not addressed in this manual, please contact your local Charder service partner.

Intended Purpose

This medical device is designed to be used in accordance with national regulations, to measure height within specifications, for height-related usage by professionals.

Clinical Benefit

Measurement results can be used by professionals to diagnose (and monitor) height-related issues.

General Handling

Device should be placed on stable, flat, solid, non-slippery surface.

Safety Instructions

The device has an expected service life of 5 years when correctly handled, serviced, and periodically inspected in accordance with manufacturer's instructions.

Cleaning

Device surface should be cleaned using alcohol-based wipes. Corrosive cleansing liquids should not be used.

Warranty/Liability

The period of warranty shall be eighteen (18) months, beginning on the date of purchase. Please retain your receipt as proof of purchase.

No responsibility shall be accepted for damage caused through any of the following reasons: unsuitable or improper storage or use, incorrect installation or commissioning by the owner or third parties, natural wear and tear, changes or modifications, incorrect or negligent handling.

Incident Reporting

Any serious incident that has occurred in relation to the device should be reported to the manufacturer, EU representative (if device is used in EU member state), and competent authority of user/subject's member state.