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MS7900

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Guidance and manufacturer’s declaration-electromagnetic emissions		
The product is intended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The product is suitable for use in all establishments other than domestic and those directly connected to a low voltage power supply network which supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Compliance	

Guidance and manufacturer's declaration-electromagnetic immunity

The product is intended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge(ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/burst IEC 61000-4-4	±2kV for power supply lines	±2kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1kV line(s) to line(s) ±2kV line(s) to earth	±1kV line(s) to line(s) ± 2kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% UT for 0,5 cycle 0% UT for 1 cycle 70% UT(30% dip in UT) for 25cycles 0% UT for 5 s	0% UT for 0,5 cycle 0% UT for 1 cycle 70% UT(30% dip in UT) for 25cycles 0% UT for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the product requires continued operation during power mains interruptions, it is recommended that the product be powered from an uninterruptible power supply or a battery.
Power frequency(50, 60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	The product power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE UT is the a.c. mains voltage prior to application of the testlevel.

Guidance and manufacturer's declaration-electromagnetic immunity

The product is intended for use in the electromagnetic environment specified below.

The customer or the user of the product should assure that is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 KHz to 80 MHz	3 Vrms 150 KHz to 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of the product including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	6 V in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	6 V in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800MHz to 2,7GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range.^b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE2

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the product is used exceeds the applicable RF compliance level above, the product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the product.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Recommended separation distance between portable and mobile RF communications equipment and the product			
The product is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the product can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the product as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,7 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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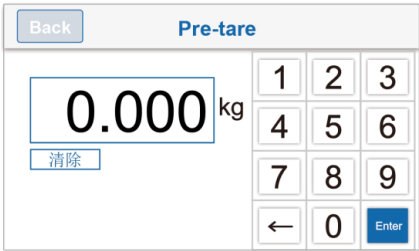
敘述	顯示器
按 [PRE-TARE] 鍵。 從最左側的數位開始，輸入預扣重量值	 The image shows a scale display in 'Pre-tare' mode. At the top left is a 'Back' button. The title 'Pre-tare' is in blue. The main display shows '0.000' in large black digits with 'kg' to the right. Below the display is a '清除' (Clear) button. To the right is a numeric keypad with digits 1-9, 0, and an 'Enter' button. A left arrow button is also present.
用數位鍵 0-9 輸入預扣重值 例：預扣值 5.0 kg，請按 0-5-0-0-0 例：預扣值 13.5 kg，請按 1-3-5-0-0 按 [Enter] 鍵確認預扣重值	 The image shows two sequential states of the scale display in 'Pre-tare' mode. The top part shows the initial state with '0-.---' on the display and a '清除' button below it. The bottom part shows the state after entering '5.000', with the display showing '5.000 kg' and the '清除' button still present. The interface elements (Back button, title, keypad, Enter button) are consistent with the previous image.

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
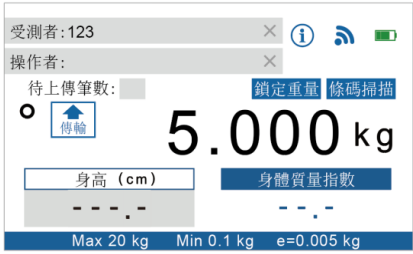
敘述	顯示器
輸入嬰兒的 ID，並按 [Hold] 鍵啟動 Hold 功能	 The scale display shows '受測者: 123' (Subject: 123) and '操作者:' (Operator:). The weight is '0.000 kg'. A red box highlights '鎖定重量' (Lock Weight). Other buttons include '待上傳筆數:' (Number of items to upload), '身高 (cm)' (Height), '身體質量指數' (BMI), 'Max 20 kg', 'Min 0.1 kg', and 'e=0.005 kg'.
將嬰兒放置于秤臺上，待重量鎖定 (例：嬰兒授乳前重量 5.000kg)	 The scale display shows '受測者: 123' (Subject: 123) and '操作者:' (Operator:). The weight is '5.000 kg'. A red box highlights '鎖定重量' (Lock Weight). Other buttons include '待上傳筆數:' (Number of items to upload), '身高 (cm)' (Height), '身體質量指數' (BMI), 'Max 20 kg', 'Min 0.1 kg', and 'e=0.005 kg'.
按[MEMORY] 鍵以儲存嬰兒授乳前的重量	 The scale display shows '受測者: 123' (Subject: 123) and '操作者:' (Operator:). The weight is '5.000 kg'. A red box highlights '授乳前重量(kg)' (Pre-feeding weight). Other buttons include '待上傳筆數:' (Number of items to upload), '身高 (cm)' (Height), '身體質量指數' (BMI), '授乳量(g)' (Feeding amount), 'Max 20 kg', 'Min 0.1 kg', and 'e=0.005 kg'.

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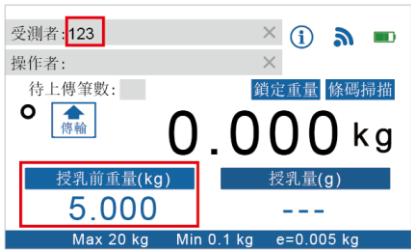敘述	顯示器
輸入嬰兒的 ID，畫面將自動出現先前儲存的授乳前重量 *此時若需要清除已儲存的授乳前重量，請長按[TARE/ZERO] 鍵 3 秒	 The scale display shows the subject ID '123' and operator 'X'. The current weight is 0.000 kg. The pre-feeding weight (授乳前重量(kg)) is 5.000 kg. The feeding amount (授乳量(g)) is 0.000 g. The scale has a maximum capacity of 20 kg, a minimum of 0.1 kg, and an accuracy of 0.005 kg.
按 [Hold] 鍵啟動 Hold 功能，並將授乳後的嬰兒放置于秤臺上，待重量鎖定 (例：嬰兒授乳後重量 5.200kg) 畫面即自動計算出授乳量(例: 200g)	 The scale display shows the subject ID '123' and operator 'X'. The current weight is 5.200 kg. The pre-feeding weight (授乳前重量(kg)) is 5.000 kg. The feeding amount (授乳量(g)) is 200 g. The scale has a maximum capacity of 20 kg, a minimum of 0.1 kg, and an accuracy of 0.005 kg.
按[Transfer] 鍵即可無線傳輸嬰兒授乳後的重量與授乳量	 The scale display shows the subject ID '123' and operator 'X'. The current weight is 5.200 kg. The pre-feeding weight (授乳前重量(kg)) is 5.000 kg. The feeding amount (授乳量(g)) is 200 g. The scale has a maximum capacity of 20 kg, a minimum of 0.1 kg, and an accuracy of 0.005 kg. The Transfer button (傳輸) is highlighted with a red box.

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EXIT General			
Auto Off Time	180s	G-Compensation	-----
Backlight	Mid	URL Host	-----
Buzzer	On	Auto Hold	On
Optional Settings	-----	Auto Transfer	Off
Date/Time	-----		
Wifi Setting	-----		

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Copyright© Charder Electronic Co., Ltd. 版權所有。