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
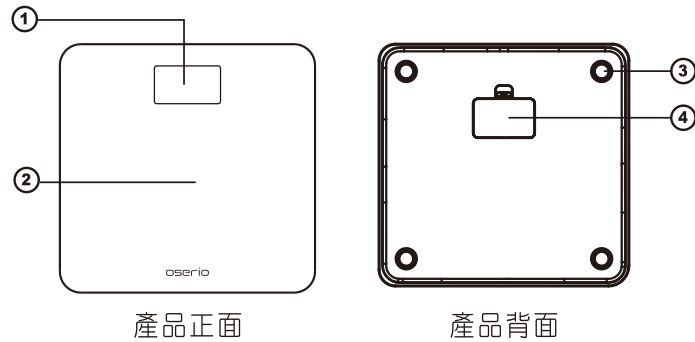
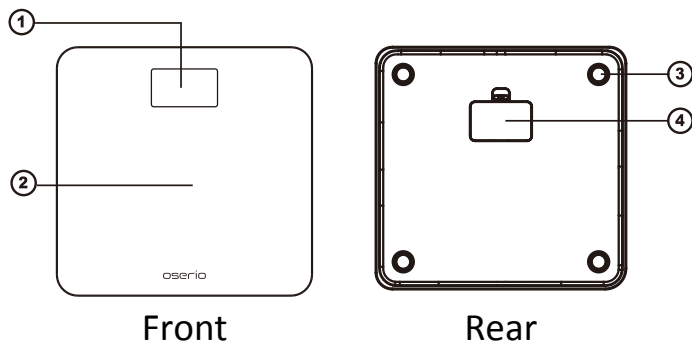
Overview


- ① LCD display ② Platform ③ Feet ④ Battery Cover

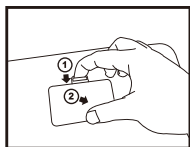
Battery Installation

- ① Pull the tab

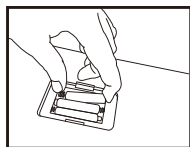- ② Remove the battery cover

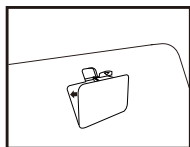- Install 3 "AAA" size
batteries properly

- Place the battery cover
back until clicks

NOTE: Be sure that the polarity of the battery is inserted right.

Important Safety Notes

- Keep the scale out of the reach of young children.
- Do not jump onto the scale.
- Senior or disabled should be assisted by others or use handrails when using the scale.
- Do not dismantle, modify or disassemble the structure of the scale or display.
- Please use the specific batteries for this scale. Do not insert the batteries with the polarities in the wrong direction.
- If the scale is not used approximately three months or more, please remove batteries from the scale.
- If no display data showing, check the batteries
- To enhance the device's using life, please do not install new and used batteries or different types of batteries together.
- Do not use the scale when the surface of the scale or the feet are wet.
- Do not place this scale in highly humid environments, near heat sources or below air conditioners, and avoid direct sunlight.
- Do not drop, vibrate, or apply strong shock on the scale
- Do not wash the scale with water and do not wipe the scale with benzene, gasoline, paint thinner, alcohol, or other volatile solvents.
- Do not place the scale on a cushioned floor surface like on a carpet or a mat. Correct measurement may not be possible.
- This scale is designed to be used indoors only (home-use).

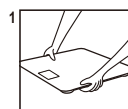
© This product is not medical device, the measurement data is for reference only.
Please consult doctor if you have any questions.

Self-calibration

After installing batteries or moving the scale, please follow the steps before weighing.


Place the scale
on a flat floor.

Step on the scale.

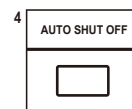
Step off the scale.

Ready to
measure weight.

Weighing reading doesn't have to be locked.

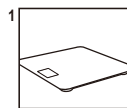Operating Instructions


1 Place the scale on a flat floor.

2 Step on the scale.

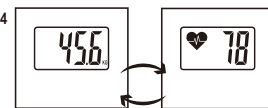3 Stand still for 12~15 seconds.

4 Measurement results showing weight ↔ heart rate

© Please re-calibrate the scale if the weight reading is incorrect.

Error Message List

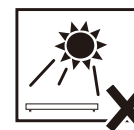
Low battery indication
-->change the batteries

Overloading or loading is
imbalanced--> stand still

Care and Maintenance

- Store the unit in a dry location.
- Clean the scale with alcohol-wiped cloth.
- Scratching/ water infiltration the scale would shorten scale lifecycle.
- Do not step on the edge of the platform or on the display.
- Place the scale free from direct sunlight.

Product Specification

Model	BHG-208 series
Capacity	180kg
Graduation	0.1kg
Power Supply	3 AAA batteries
Weight	Approx. 1.6kg
External Dimensions	Main unit: 300mm(w) x 290mm(d) x 25mm(h) LCD display: 76mm(w) x 42.5mm(h)
Operating Environment Limit	Temperature 5°C to 35°C Humidity 30% to 85% RH
Storage Environment Limit	Temperature -20°C to 60°C Humidity 10% to 95% RH
Weight Accuracy	5~180kg : ±1%
Heart rate by ballistocardiography (BCG)	30~250 pulse / min

Importer : Charder Electronic Co., Ltd.
No.103, Guozhong Rd., Dali Dist., Taichung City 412, Taiwan (R.O.C.)

Manufacturer : Krell Precision (Yang Zhou) Co., Ltd.
No.28 XingYang Road, Development Zone, YangZhou JiangSu 225009, China

www.oserio.com