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
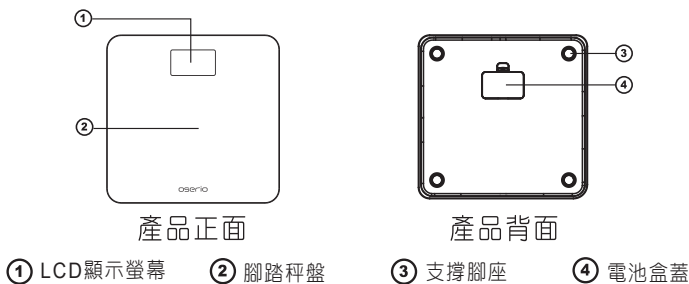
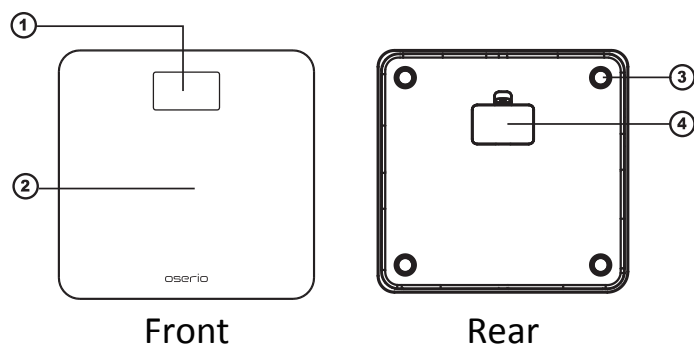
Overview

- ① LCD display ② Platform ③ Feet ④ Battery Cover

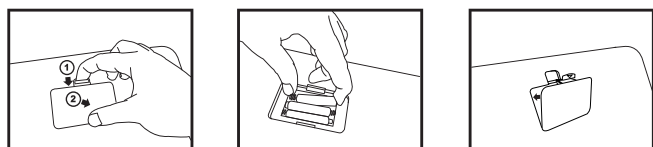Battery Installation

- ① Pull the tab
② Remove the battery cover

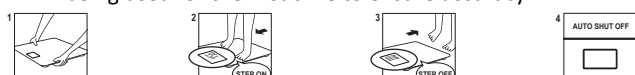
Install 3 "AAA" size
batteries properly

Replace battery cover
(a clicking noise will be heard)

NOTE: Ensure batteries are inserted correctly (according to polarity)

Self-calibration

The product may require a quick self-calibration before being used for the first time to ensure accuracy.

- Place product on stable, flat, solid, non-slippery surface (no carpet).
- Step on measurement platform. Weight will be displayed on screen.
- Step off from measurement platform. After product turns off automatically after a short period of time, self-calibration is complete.
- Ready to measure weight.

Weighing reading doesn't have to be locked.

☉ Please re-calibrate the scale if the weight reading is incorrect.

Important Safety Notes

General Handling

- Product should be placed on stable, flat, solid, non-slippery surface.
- Place the product in an area free from direct sunlight, heating equipment, high humidity, or extreme temperature change.
- Never submerge in water.
- Usage on soft surfaces (ex: carpet) may result in inaccurate results.

Disposal

- All batteries contain toxic compounds; batteries should be disposed of via designated competent organizations. Batteries should not be incinerated.
- This product is not to be treated as regular household waste, but should be taken to a designated collection points for electronics.
- Further information should be provided by local waste disposal authorities.

☉ The BHG-208BT is not a diagnostic product. Measurement data is for reference only.

Operating Instructions

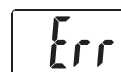
- Place the scale on a flat floor.
- If the O'Care App has not been downloaded to your smartphone, the download link can be found on the bottom label. After installing the App, please turn on Location and Bluetooth, and register an account. After account has been registered, please ensure O'Care App is active before conducting measurement.
- Step on the scale. Scale will display weight measurement.
- After weight measurement is complete, scale will measure heart rate (~15 seconds). Stand still (refrain from talking) until measurement is complete.
- After measurement is complete, weight and heart rate will be displayed on the screen.
- If the O'Care App is turned on and active, results will be automatically synced to your smartphone.

Error Messages


Low battery indication
--> change the batteries

Overloading or loading is
imbalanced--> stand still

Use and Storage

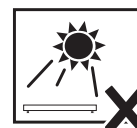- Product should be stored in clean, cool, dry location when not in use, away from direct sunlight and extreme temperatures.
- This product is not a toy. Children should not be allowed to play with it.
- Avoid corrosive liquids and materials. Do not use detergents or cleaners to clean the product. This may damage the coating protecting the electrodes used for body composition measurement.
- Wipe the platform and display using clean soft cloth. Avoid rough, sticky cloth, to prevent scratching the measurement platform and screen panels.

Product Information

Model	BHG-208BT series
Capacity	180kg
Graduation	0.1kg
Power Supply	3 AAA batteries
Weight	Approx. 1.6kg
External Dimensions	Main unit: 300mm(w) x 290mm(d) x 25mm(h) LCD display: 76mm(w) x 42.5mm(h)
Operating Environment Limit	Temperature 5°C to 35°C Humidity 30% to 85% RH
Storage Environment Limit	Temperature -20°C to 60°C Humidity 10% to 95% RH
Weight Accuracy	5~180kg : ±1%
Heart rate by ballistocardiography (BCG)	30~250 pulse / min

Importer : Charder Electronic Co., Ltd.

No.103, Guozhong Rd., Dali Dist., Taichung City 412, Taiwan (R.O.C.)

Manufacturer : Krell Precision (Yang Zhou) Co., Ltd.

No.28 XingYang Road, Development Zone, YangZhou JiangSu 225009, China

www.oserio.com