


Android



iOS


Overview

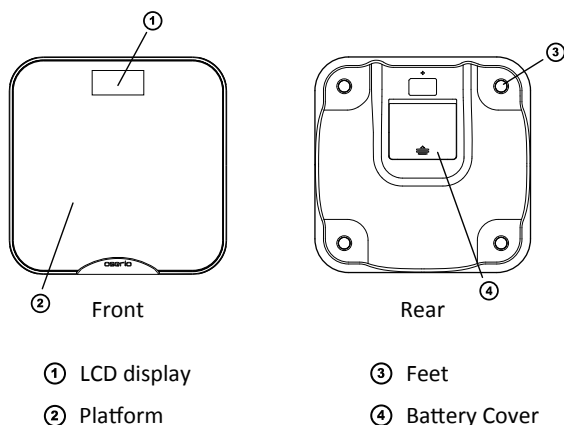


Reminder :

- Please follow the **connecting instructions** to start using it .

Battery Installation

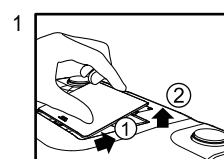

Open the battery cover

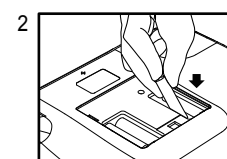


Install 4 "AA" size batteries properly

- ① Slide the battery cover
- ② Lift the cover up

NOTE:

- Be sure that the polarity of the battery is set properly.

Operating Instructions

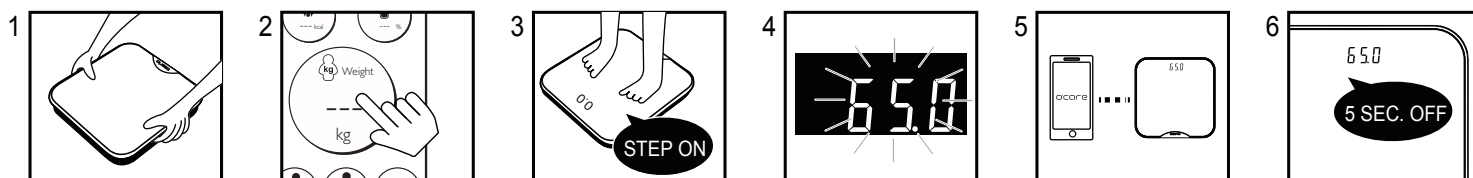

- 1 Place the unit on a hard, flat surface .
- 2 Run o'care APP and enter "kg" bubble .
- 3 Step on the scale .
- 4 Complete weighing .
- 5 Connecting with APP .
- 6 Turning off automatically after transmitting .


NOTE:

- The unit will shut down automatically if the weighing data can't be transmitted in 30 seconds .

Error Message List



Replace batteries immediately while "LO" appears.

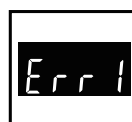

Connecting error.
Please try again or contact our service line .






Overloading or loading is unbalanced within 1 minute.
Please stand still during measurement.



Connecting error.
Please try again or contact our service line .



Important Safety Notes

1. Keep out of reach of young children.
2. Do not jump on the unit platform.
3. Do not step on the edge of unit platform.
4. Person with disabilities or physically frail should be assisted by another person when using this unit.
5. Do not disassemble or modify the unit.
6. Using the batteries specify by the product specifications, polarities match the polarities of the battery compartment as indicated.
7. Remove the batteries if the unit not be used for three months or longer.
8. If the display not shows, check the batteries.
9. Do not mix used and new batteries or different types of batteries to extend the life of the unit.
10. Do not use the unit on slippery surfaces such as tile floors or wet floors.
11. Place the unit in an area free from direct sunlight, heating equipment, high humidity or extreme temperature change.
12. Avoid excessive impact or vibration to the unit.
13. Never submerge in water. Use alcohol to clean the electrodes and glass cleaner (applied to a cloth first) to keep them shiny.
14. Place the unit on a hard, flat surface where there is minimal vibration to ensure safe and accurate measurement.
15. Always remove your shoes before stepping on the measuring platform.
16. Properly dispose of batteries according to applicable local regulations.
17. This unit is intended for home use only. It is not intended for professional use in hospitals or other medical facilities.

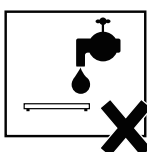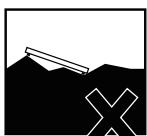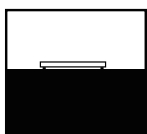

Care and Maintenance

1. Store the unit in a safe and dry location.
2. Avoid place on the aisle, to avoid accidentally tripped over pedestrians.
3. Never submerge in water.
4. Do not expose the unit to direct sunlight, extreme hot or cold temperatures or high humidity.
5. Avoid contact with corrosive chemicals or gas.
6. Clean the unit with a soft dry cloth for daily use.
7. Wipe the platform and display using clean soft cloth.
8. Do not use benzene, detergents or gasoline to clean the unit.
9. Avoid rough, sticking cloth, to prevent scratching the platform and screen panels.
10. Prevent water infiltration affecting the measurement accuracy.

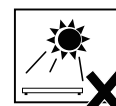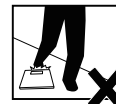

Product Specification

Power Supply	4 AA batteries	Weight	Approx. 2 kg
External Dimensions	Main unit: 320mm(d) x 320mm(w) x 31mm(h)		
Operating Environment Limit	Temperature 5°C to 35°C Humidity 30% to 85% RH		
Storage Environment Limit	Temperature -20°C to 60°C Humidity 10% to 95% RH		
Weight Accuracy	5 ~ 200kg : ± 1% / 11 ~ 440lb : ± 1% + 0.1lb		
Display Content	Weight: 5 ~ 200kg	Graduation: 0.1 kg	
	BMI: 0.0 ~ 200.0	Graduation: 0.1 kg	
	Weight: 11 ~ 440lb	Graduation: 0.1 kg	
	BMI: 0.0 ~ 200.0	Graduation: 0.1 kg	
Accessories	Manual (Warranty card) , Connecting instructions		



oserio®
Health is Beautiful

Warranty Card

Customer name : _____

Purchase date : _____

Model No. : _____

Serial No. : _____

1 year warranty

Dealer Chop: _____

Term&Condition

1. oserio warrants that this product is free from material and manufacturing defects for a period of one year from date of retail purchase.

If product proves to be defective in material or manufacturing during the warranty period, oserio will repair or replace this product, free of charge.

2. If you have any questions regarding this product or warranty, please contact your dealer or Customer Service. If product needs to be returned, a completed warranty card filled out by the dealer confirming date of purchase is necessary.

3. The warranty does not cover normal wear & tear, or malfunction caused by misuse, damage, or non-compliance with instruction manual.