


o'care

User Manual

MEG-213

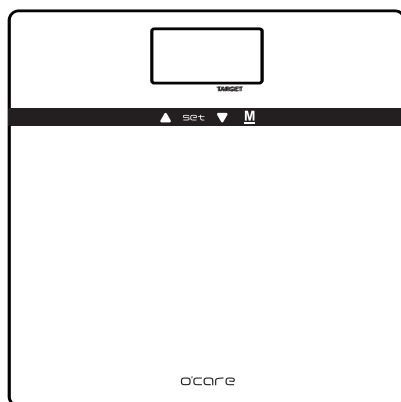

Please keep the instruction manual at hand all the time for future reference.

Table of Contents

<i>Important Safety Notes</i>	2
<i>Specifications</i>	3
<i>Overview</i>	4
<i>Battery Installation</i>	5
<i>Self-Calibration</i>	6
<i>Operating Flow Chart</i>	7
• 1-A: Weight Mode	8
• 1-B: BMI / Body Water / BMR / Bone Mass Mode	9
• 1-C: Target Weight Mode	10
• 1-D: Mommy / Baby Mode	11
<i>Memory Mode</i>	12
<i>Error Message List</i>	13
<i>Care and Maintenance</i>	14


Important Safety Notes

- Keep the scale out of the reach of young children.
- Do not jump onto the scale.
- Do not step on the edge of the platform or on the display.
- Senior or difficultly abled should be assisted by others or use handrails when using the scale.
- Do not dismantle, modify or disassemble the structure of the scale or display.
- Please use the specified batteries for this scale. Do not insert the batteries with the polarities in the wrong direction.
- If the scale is not used approximately three months or more, please remove batteries from the scale.
- If there is no display on the screen after turning on, please check if batteries have run down or have been installed in.
- To enhance the device's using life, please do not install new and old batteries or different types of batteries together.
- Do not use the scale when the surface of the scale or the feet are wet.
- Do not place this scale in highly humid environments, near heat sources or below air conditioners, and avoid direct sunlight.
- As this scale is a precision instrument, do not drop, vibrate, or apply strong shock.
- Do not wash the scale with water and do not wipe the scale with benzene, gasoline, paint thinner, alcohol, or other volatile solvents.




Important Safety Notes

- Do not place the scale on a cushioned floor surface like on a carpet or a mat. Correct measurement may not be possible.
- Please use the scale with bare foot.
- Disposal of used batteries should be complied with the national regulation for electronic devices.
- Do not use mobile phone near the scale.

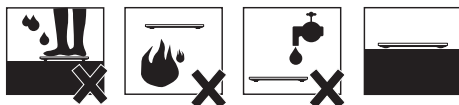

Specifications

Product	O'CARE MEG-213				
Capacity	180 kg	Graduation	0.1 kg	Accuracy	± 1%
Dimensions	Main Unit: 330(d)x330(w)x30(h) mm LCD: 92(d)x45(w) mm				
Power Supply	4 AA Batteries				
Standard Accessories	Manual & Warranty Card				
Operating Temperature & Humidity	Temperature 5 °C to 40 °C Humidity 30% to 85% RH				
Storage Temperature & Humidity	Temperature -20 °C to 60 °C Humidity 10% to 95% RH				

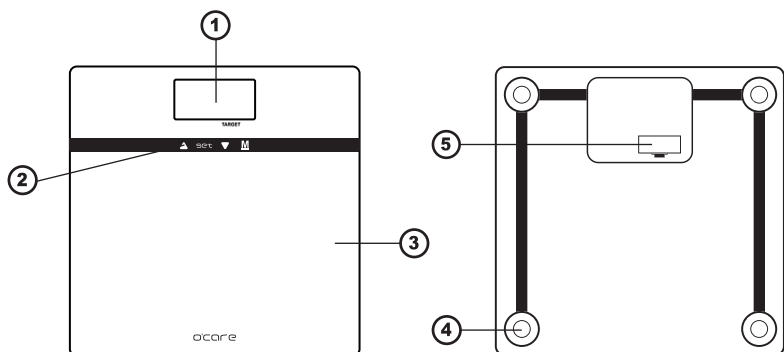


- ① LCD display
- ② Setting buttons
- ③ Platform
- ④ Feet
- ⑤ Battery Cover

NOTE:


Press the button with your finger without gloves and humidity.
Keep metal away from the buttons.

Battery Installation

Please verify if the batteries are installed correctly.

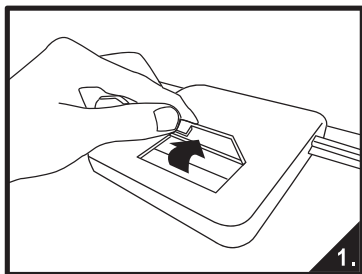

Open the battery cover

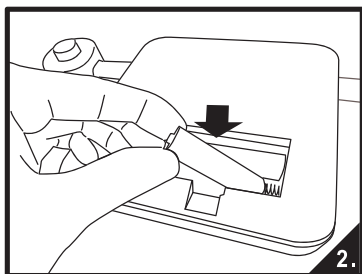

Install 4 "AA" size batteries

NOTE:


Measurements and user profiles will not be erased during battery replacement.

Be sure that the polarity of the battery is set properly.


Self-Calibration




After installing batteries or moving the scale, please follow the steps before weighing.

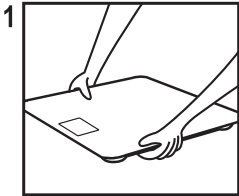


Place the scale on the flat floor.

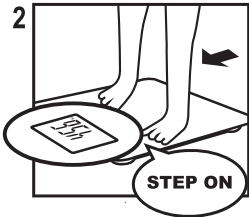

Step on the scale.
(Weighing reading doesn't have to be locked)


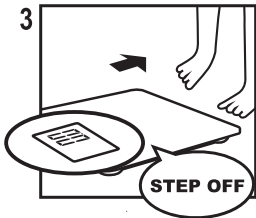


Step off the scale.

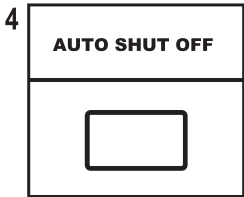

Ready to measure weight.

Operating Flow Chart

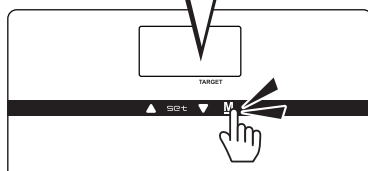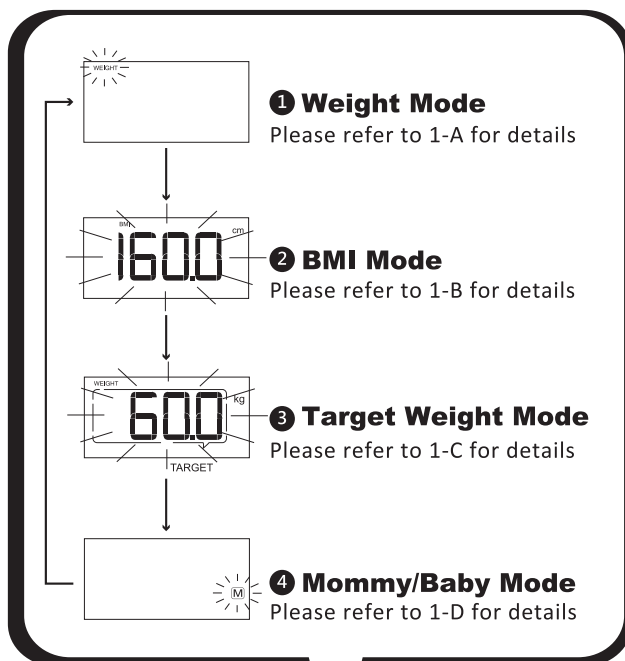


Press **M** to switch modes




This model will memorize the last chosen mode after turning off. If you would like to use the last chosen mode, please step on the scale and no need to press any keys.

1-A: Weight Mode

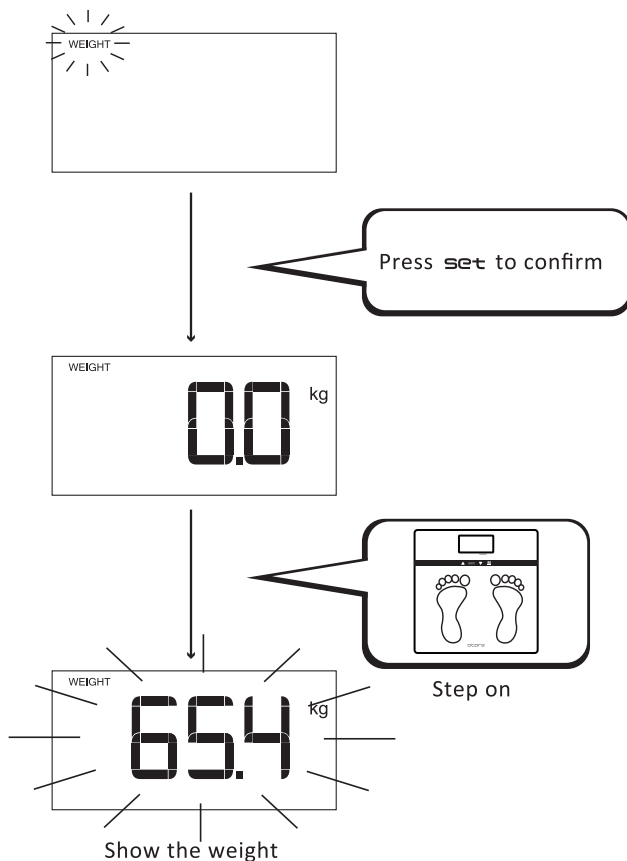


The measuring result will be read out and turn off automatically.

1-B: BMI / Body Water / BMR / Bone Mass Mode

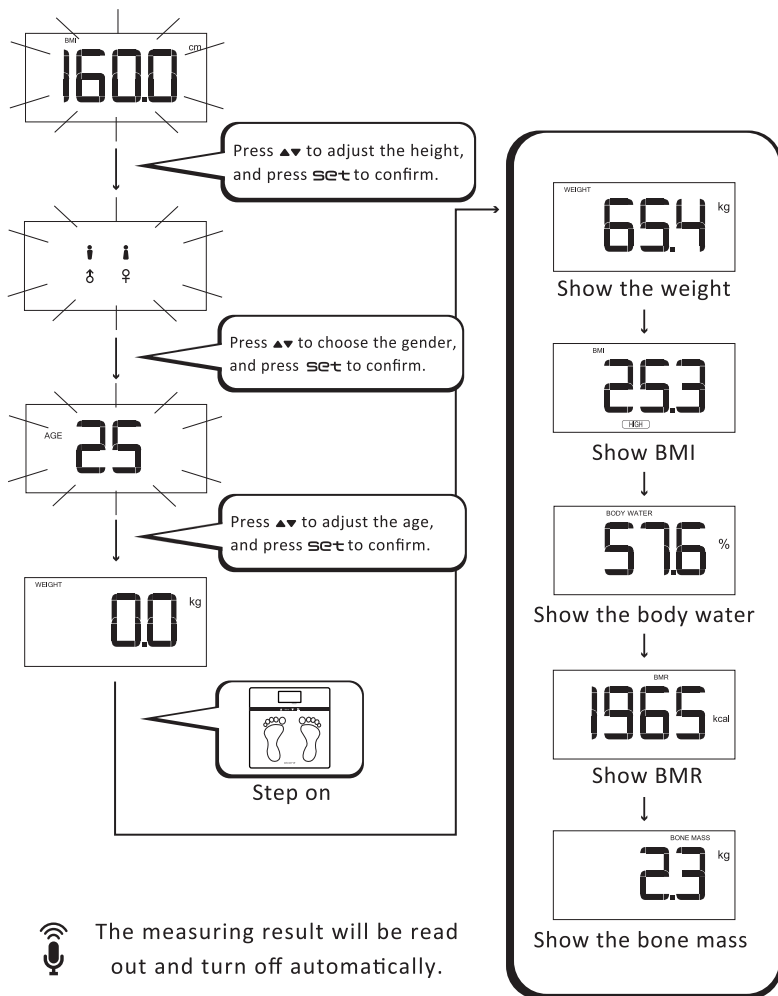

1-C: Target Weight Mode

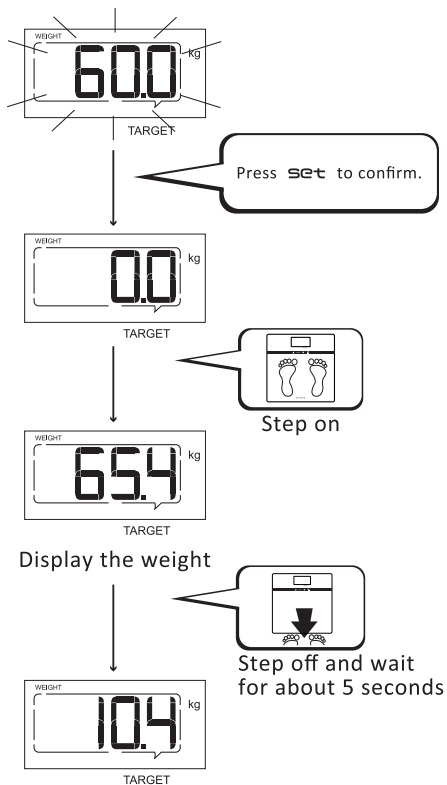


Display the weight

Show the difference between the target weight and current weight.




The measuring result will be read out and turn off automatically.

1-D: Mommy / Baby Mode

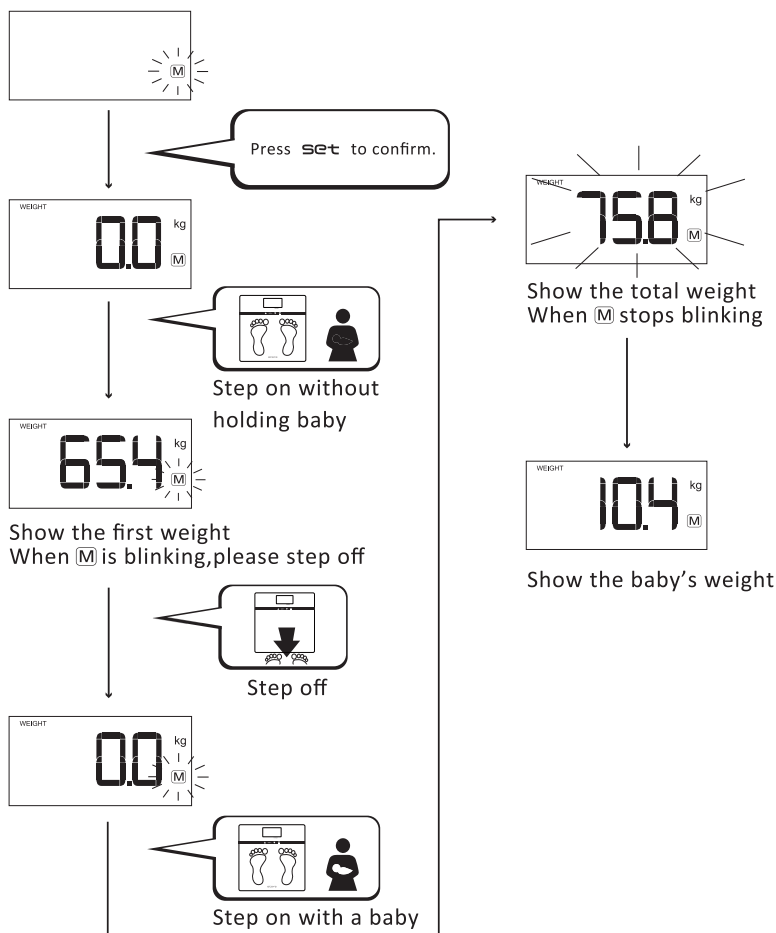

The measuring result will be read out and turn off automatically.

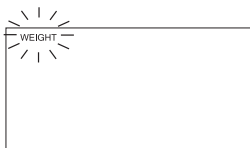

Press ▲▼ to review weighing history records. If there is no history record, pressing ▲▼ won't show any weighing records.



Press ▲▼ to track weighing history records.



The measuring result will be read out and turn off automatically.



Error Message List




Low battery indication

→ change the batteries.



Overloading or loading is imbalanced

→ stand still.

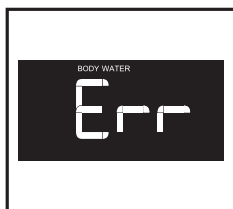

The value of body water is beyond the product specification.



EEPROM is inactive

→ reset user profiles or contact the local service.

NOTE:

When an error message displays, the scale will not read out.

1. Store the unit in a dry location.
2. Clean the scale with alcohol-wiped cloth.
3. Scratching/ water infiltration the scale would shorten scale lifecycle.
4. Do not step on the edge of the platform or on the display.
5. Place the scale free from direct sunlight.

NOTE

NOTE



Manufacture by Krell Precision (Yangzhou) Co., Ltd.
No.28, Xing Yang Rd., Economic Development Zone, Yang Zhou
City, Jiang Su Province, China

For Charder Electronic Co., Ltd.
No.103, Guozhong Rd., Dali Dist., Taichung City 41262, Taiwan

Distributed PT. Next Level Medical
JL. Bukit Gading Raya, Komp. Gading Bukit Indah Blok TB No. 30,
Kelapa Gading Barat, Jakarta Utara, 14240 Indonesia