



www.oserio.com

Company: Charder Electronic Co., Ltd. Service Line: +886 4 24079943 Made in China

Address: 103 Guozhong Rd. Dayuan Vil. Dali District. Taichung City. Taiwan

130-BYG209-101

oserio[®] Electronic Weighing Scale
Health is Beautiful

oserio Electronic Weighing Scale Instruction Manual – BYG-209

Thank you for selecting oserio series of Health Management Products.

This Instruction Manual will guide you through the setup procedures and outline the key features.

1

Before You Use The Scale

1.1 Know Your Unit	1
1.2 Battery Installation	1
1.3 Product Features	2
1.4 Important Safety Notes	3

2

Operation Instructions


2.1 Taking Weight	5
-------------------	---

3

More Information

3.1 Care and Maintenance	6
3.2 Specifications	7


Content

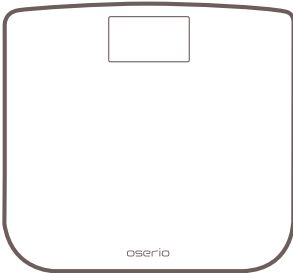


BYG-209

Before You Use The Scale


1.1 Know Your Unit

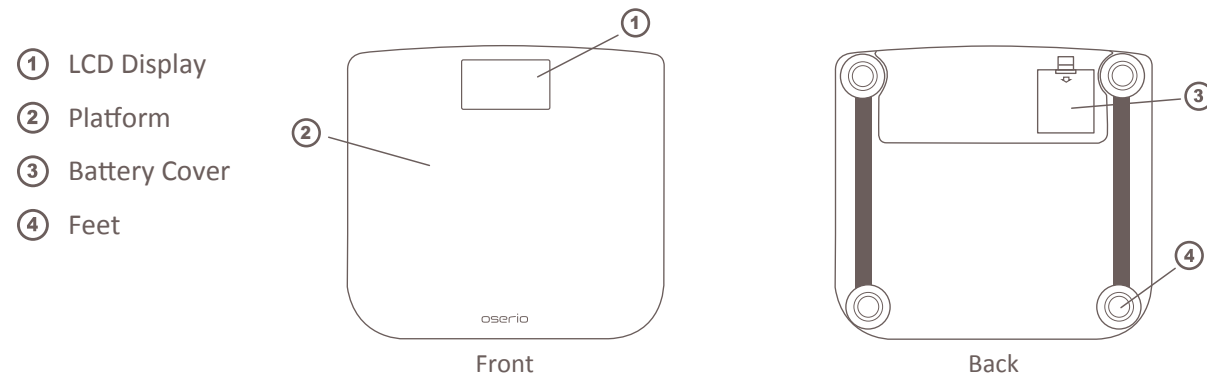

1.2 Battery Installation

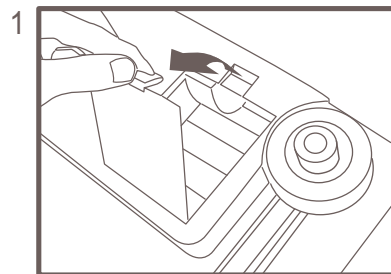

Open the
battery cover

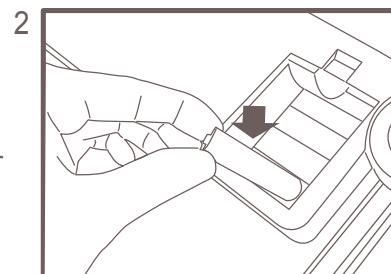

Install 4 AA batteries

Before You Use The Scale


1.3 Product Features

- ⦿ Modern styling of 6 mm toughened glass plate
- ⦿ Ultra slim and low profile digital scale
- ⦿ Extra large white backlit and easy- to- read LCD display
- ⦿ Tap on activation
- ⦿ Auto-off for long battery life

1.4 Important Safety Notes


Caution:

1. Keep the scale away from young children.
2. Do not jump on the platform.
3. Do not step on the edge of the unit and display.
4. Person with disabilities or physically frail should be assisted by another person when using this unit.
5. Do not open, repair or disassemble the unit or display.
6. Please use the specified battery (CR2032), and observing correct polarity.
7. Remove battery if you do not use this device for a long period of time.

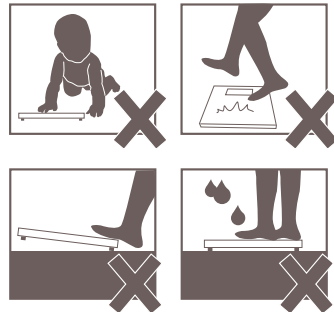

1.4 Important Safety Notes


8. Do not stand on the scale with wet feet and do not step on the scale when its surface is wet- danger of slipping!
9. If nothing is displayed on the LCD, check the battery.
10. Keep this device refrain from water, moisture, chemicals, corrosive gases, direct sunlight and heat.
11. Avoid impact, severe shock or vibration.
12. Properly dispose of batteries according to applicable local regulations.
13. Place the scales on a firm level floor; a firm floor covering is required for correct measurement.
14. Take off your shoes before using the scale.
15. Do not use this unit while operation transmitters (such as a cell phone) as this may affect your results.

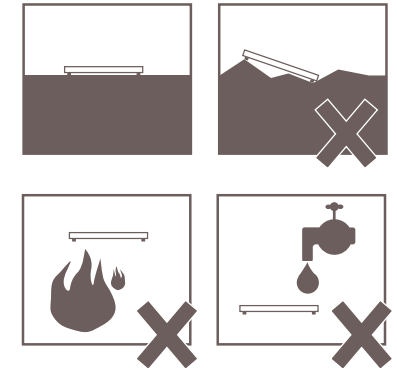

2


Operating Instructions

2.1 Taking Weight

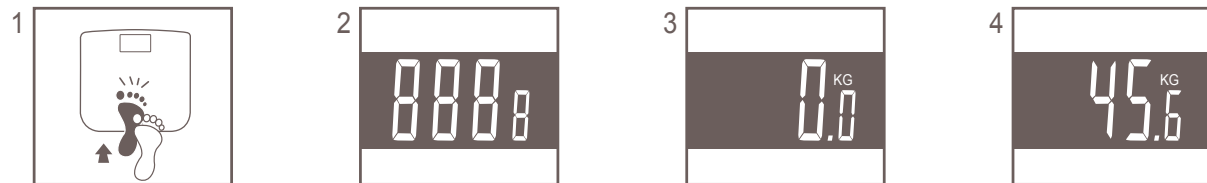

(Please do not step on the scale)

Step on the scale to measure your weight and stand still with your weight distributed evenly on both legs.

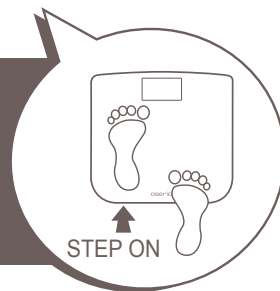

1. Tap on the platform forcefully to start
2. Running calibration (Please do not step on the scale)
3. While scale is ready to measure, the display will appear “ 0.0kg ”
4. Displaying your weight

★ When you step off the scale will switch off after 10 seconds

3

More Information

3.1 Care and Maintenance

1. Never submerge the unit in water.
2. The dry and soft towel is recommended for cleaning the unit. Wring out the wet towel before cleaning if necessary.
3. To prevent scratch from using the cloth to clean the plate and display.
4. Inaccuracy may cause by scale with water.

3.2 Specifications

Model	BYG-209
Power Supply	4 AA batteries
Weight	2.5 kg
Dimensions	Body: 340(L)x 315 (W) x 32 (H) mm LCD : 96 (L) x 53 (W) mm
Operation Environment Limit	Temperature: 5~35℃ Humidity: 30 ~ 85%RH
Storage Environment Limit	Temperature: -20~60℃ Humidity: 10 ~ 95%RH
Weight Accuracy	5~180 kg (±1%)
Accessories	Instruction manual

oserio®

Health conscious • Professional • Innovative