



Manufacturer's Declaration of Conformity

This product has been manufactured in accordance with the harmonized EU / UK standards, following the provisions of the below stated directives:

Electromagnetic Compatibility Directive 2014/30/EU
Electromagnetic Compatibility Regulations 2016

Low Voltage Directive 2014/35/EU
Electrical Equipment (Safety) Regulations 2016

RoHS Directive 2011/65/EU and Delegated Directive (EU) 2015/863

Digital Baby Scale CUPID 1 USER MANUAL

READ AND SAVE THESE INSTRUCTIONS

YOU ARE STRONGLY RECOMMENDED TO READ THE INSTRUCTIONS AND WARNINGS HEREIN BEFORE USING THE EQUIPMENT. THEY CONTAIN IMPORTANT INSTRUCTIONS FOR INSTALLATION, PROPER USE AND MAINTENANCE OF THE DEVICE.

BEFORE USING BABY SCALE

The electronic baby scale has been designed to help you monitoring the growth of your baby. It is able to show how much milk was given by each meal and helps you monitoring your child's progress in terms of weight gain during all section of growth.

TECHNICAL DATA

BASIC OPERATION OF THE SCALE

Capacity / Division	20kg / 10g	Press ON/OFF/TARE button to switch on and off the scale. Press ON/OFF/TARE button to TARE the scale. Switching on the scale the display shows "8888" until the display shows "0.0" is ready for use. To switch off the scale by pressing ON/OFF/TARE button, otherwise the scale is automatically power off after 60 seconds.
Key Functions	ON/OFF/TARE/UNIT /HOLD	
Units	Kg / lb	
Dimension	320*520*80mm	
Weight	2.20kg	
Storage Temperature	0~50℃	
Rel. Humidity Level	85%	
Accuracy Tolerance	± 3%	
Power Supply	9V alkaline battery	
*HOLD(weighing lock) feature *Removable Pan		

SETTING TO WORK

Place the scale on a flat and hard surface.

Installing 9V alkaline battery.

- 1.Remove the battery cover at the underside of the scale.
- 2.Insert a 9V alkaline battery in the fastening, connect cable.
- 3.Close the cover.

***RECYCLE THE BATTERY AFTER BATTERY EXHAUSTED.**

FEATURE DESCRIPTION

TARE

If you want to weigh your baby with a soft material or weighing without pan you should use tare feature to tare the weight of object before your weighing. First, power on the scale till display shows "0.00" place an object(..like a towel) on the scale and press ON/OFF/TARE button to tare the weight of object. Then weigh your baby. The result will be the net weight of the baby.

HOLD

Due to the baby's activity, the value fluctuates all the time during the weighing. To lock the weight of the baby by pressing HOLD button. The display shows "hold" and the scale automatically calculates the average of the weight it has taken. Once the result has been calculated the HOLD symbol flickering.

*To disable the HOLD feature by pressing HOLD button again.