




Indicator Stand

USER MANUAL SM2714

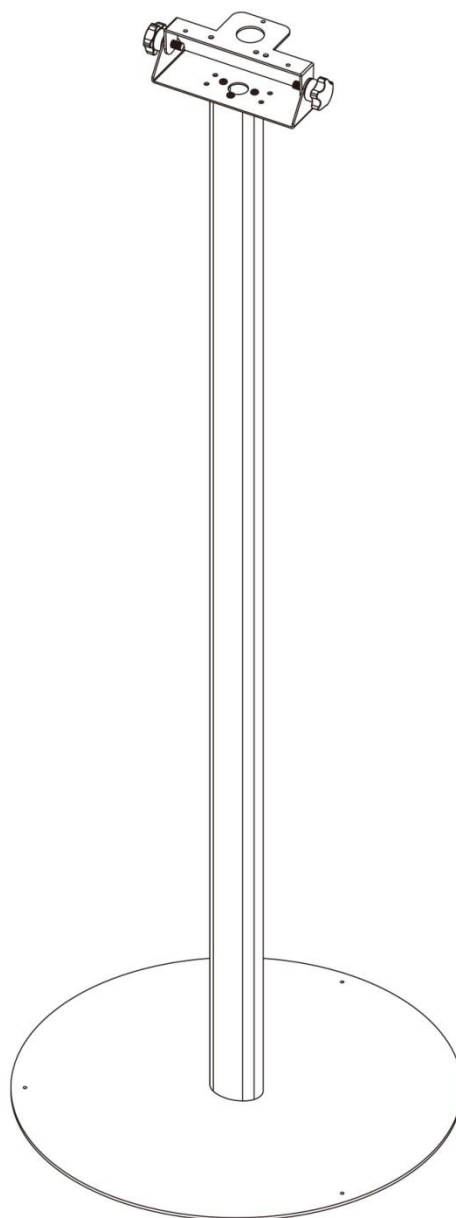

Please keep the instruction manual at hand all the time for future reference.

chorder[®]

Reliable | Precise | Easy-to-use

Copyright Notice

Charder Electronic Co., Ltd.

No.103, Guozhong Rd., Dali Dist., Taichung City 41262 Taiwan

Tel: +886-4-2406 3766

Fax: +886-4-2406 5612

Website: www.chardermedical.com

E-mail: info_cec@charder.com.tw

Copyright© Charder Electronic Co., Ltd. All rights reserved.

This user manual is protected by international copyright law. All content is licensed, and usage is subject to written authorization from Charder Electronic Co., Ltd. (hereinafter Charder) Charder is not liable for any damage caused by a failure to adhere to requirements stated in this manual. Charder reserves the right to correct misprints in the manual without prior notice, and modify the exterior of the device for quality purposes without customer consent.

Charder Electronic Co., Ltd.
No. 103, Guozhong Rd., Dali Dist.,
Taichung City, 41262 Taiwan

CONTENTS

I . SAFETY NOTES	2
II. PARTS	3
III. INDICATOR ASSEMBLY-FOR DP2701	4
IV. INDICATOR ASSEMBLY-FOR DP3400	6
V. INDICATOR ASSEMBLY-FOR DP37	8
VI. INDICATOR ASSEMBLY-FOR DP48	10
VII. ASSEMBLY-FOR THERMO PRINTER TP21 SERIES	12
IX. DECLARATION OF CONFORMITY	13

I . Safety Notes

Preface

Thank you for choosing CHARDER MEDICAL product. All features of this state of the art product were designed and are optimized for simple and straightforward use. If you have any queries or experience any problems not addressed in the operating instructions, please contact the dealer from whom you purchased the product.

General information

We strongly recommend you use the products on flat and hard surface. Any soft surface, like carpet will cause unstable.

Cleaning

We would recommend using alcohol based wipes or similar when cleaning the product. Do not use large amounts of water when cleaning the product as this will cause damage to the products. You should also refrain from using corrosive liquids or high-pressure washers.

Warranty-liability

Charder medical products sold by an authorized dealer are warranted for 2 years from the original date of purchase and only to the original end user. Charder medical products sold by an authorized dealer are warranted against manufacturer defects under normal use. The warranty does not cover the following:

Unsuitable or improper storage, installation, use or maintenance; apparent damage resulting from an accident, neglect or abuse or improper handling; damage caused by high humidity, liquid or foreign substance infiltration; damage due to use in temperatures outside the "Operating temperature" range.


Disposing of the product

This product is not to be treated as regular household waste, but should be handed in to an equipment-recycling center.


II. Parts

Stand for DP2701/DP3400/DP37/DP48 series


No.	ITEM	SPEC.	QTY
1	Screw for Indicator	SW-8060	2
2	Bracket A	SS-4961	1
3	Bracket B	SS-4971	1
4	Column	SS-3751A	1
5	Round plate	SS-6151	1
6	Rubber foot	MF-016ST	3
7	Hexagon screw	M3*6	3
8	Countersunk Hexagon	M4*0.7*12	3
9	Self tapping screw	3*8	3
10	Rivet nut	M5	2
11	Phillips head screw	M4*0.7*8	3
12	Washer head screws	M5*0.8*11	2

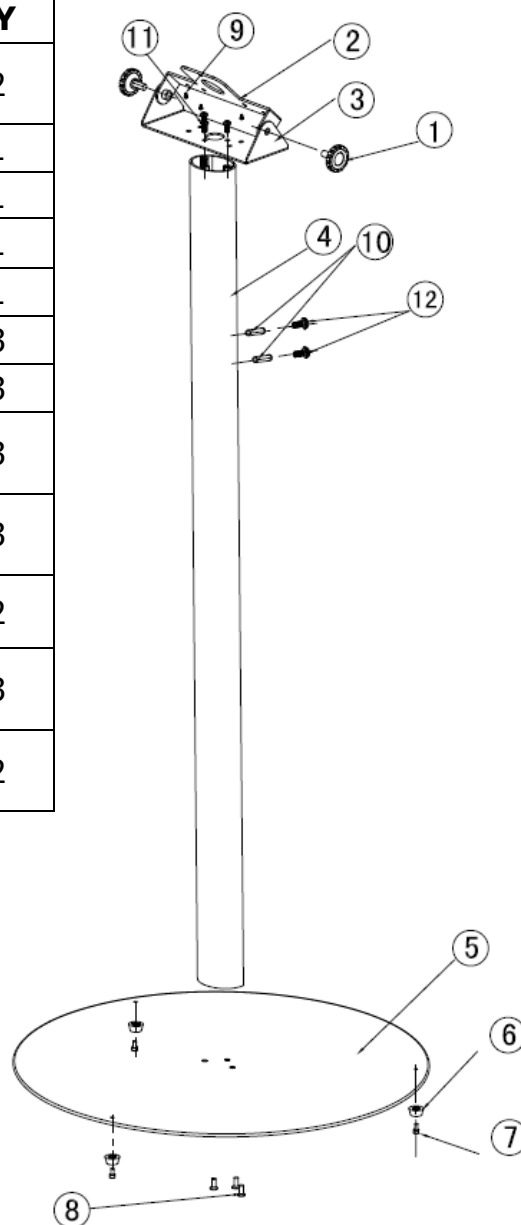


III. Indicator Assembly-for DP2701

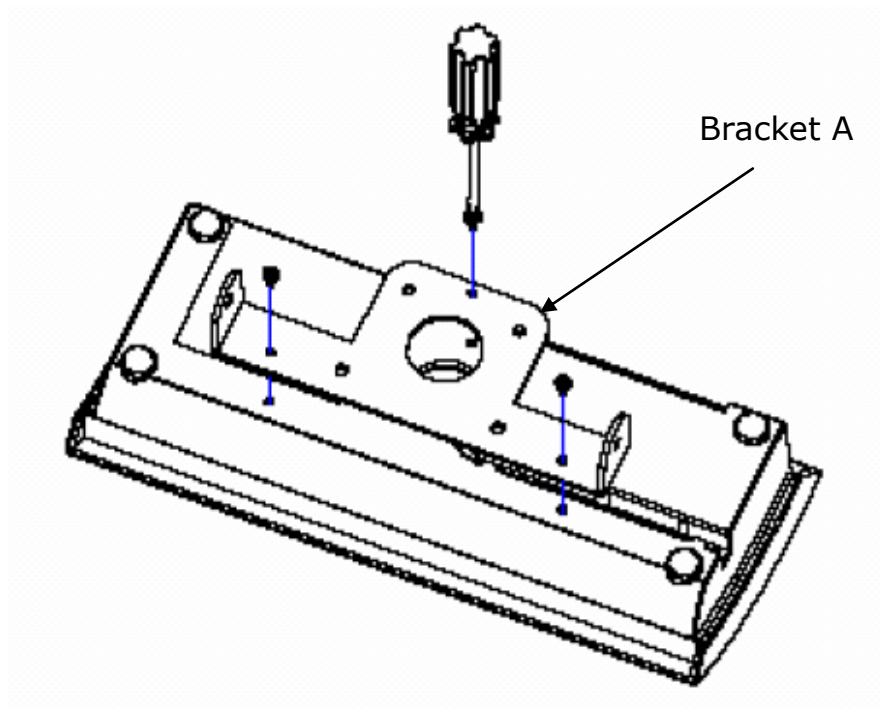

Fixing Bracket A on indicator with three screws along with indicator screw set.

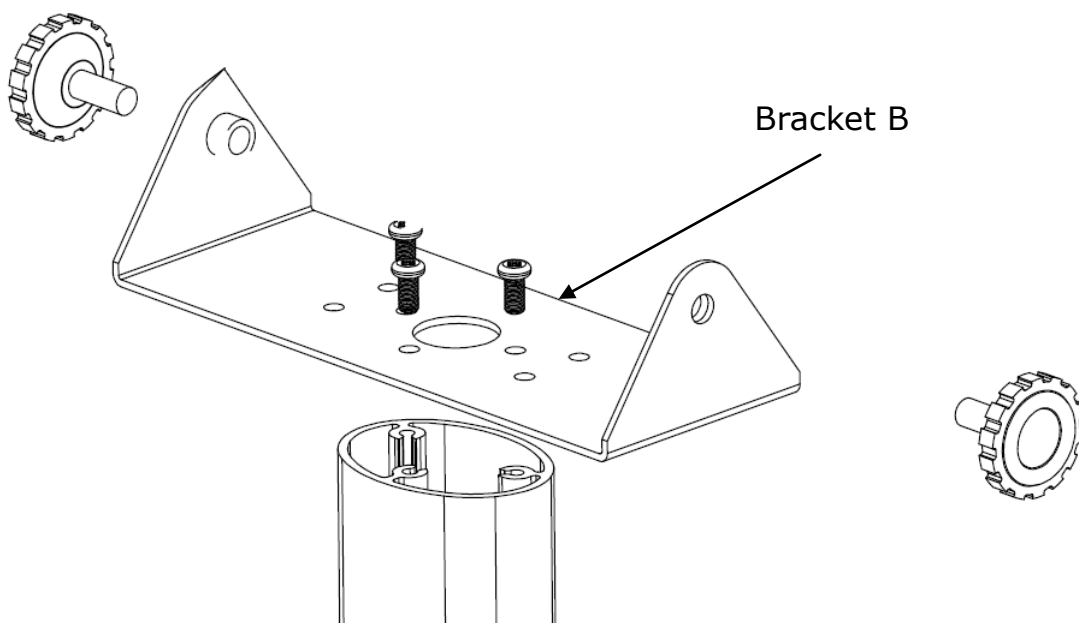


Fixing Bracket B on column with three cross screws ITEM 8.

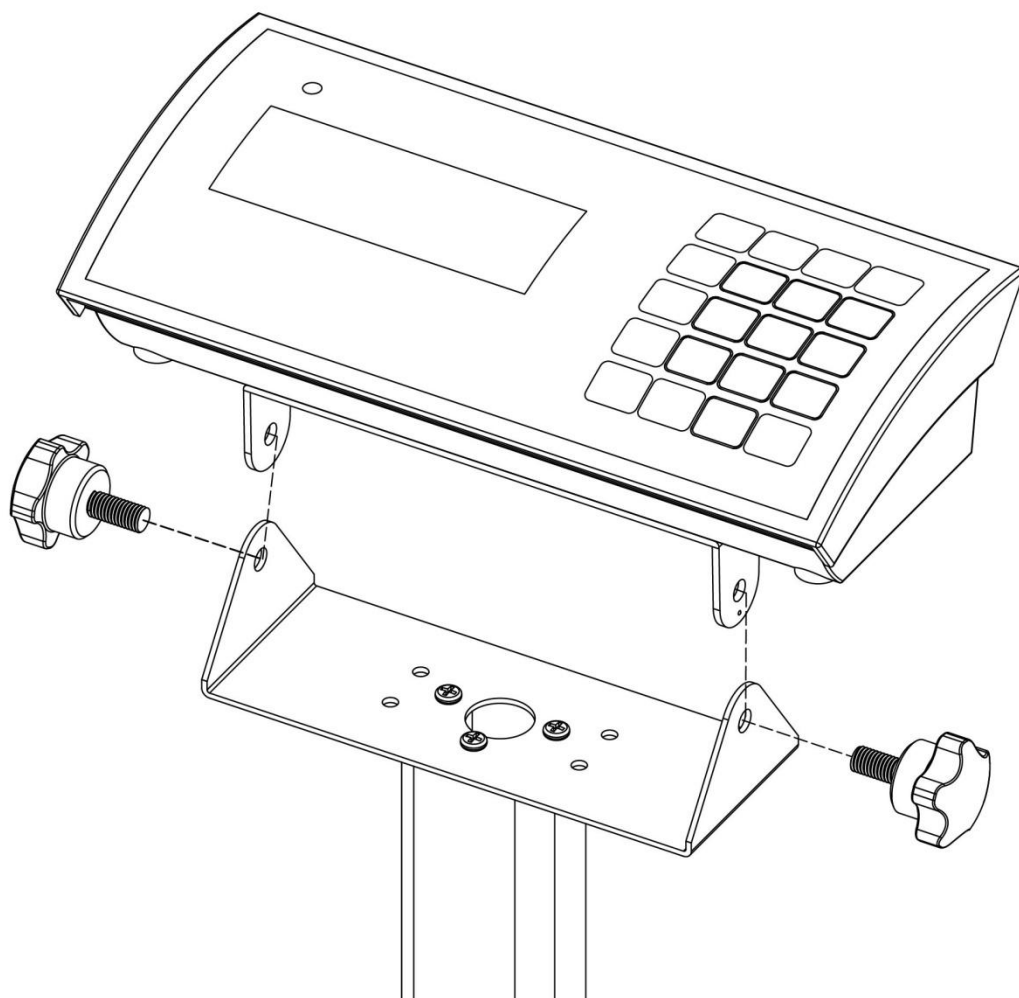


Fasten screws for indicator to complete indicator assembly.


IV. Indicator Assembly-for DP3400

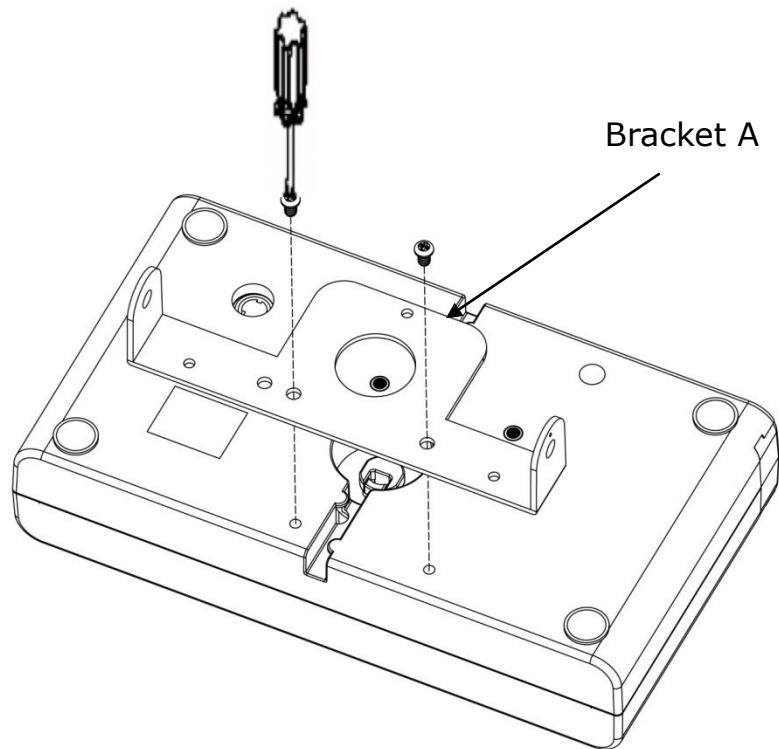

Fixing Bracket A on indicator with three screws along with indicator screw set.

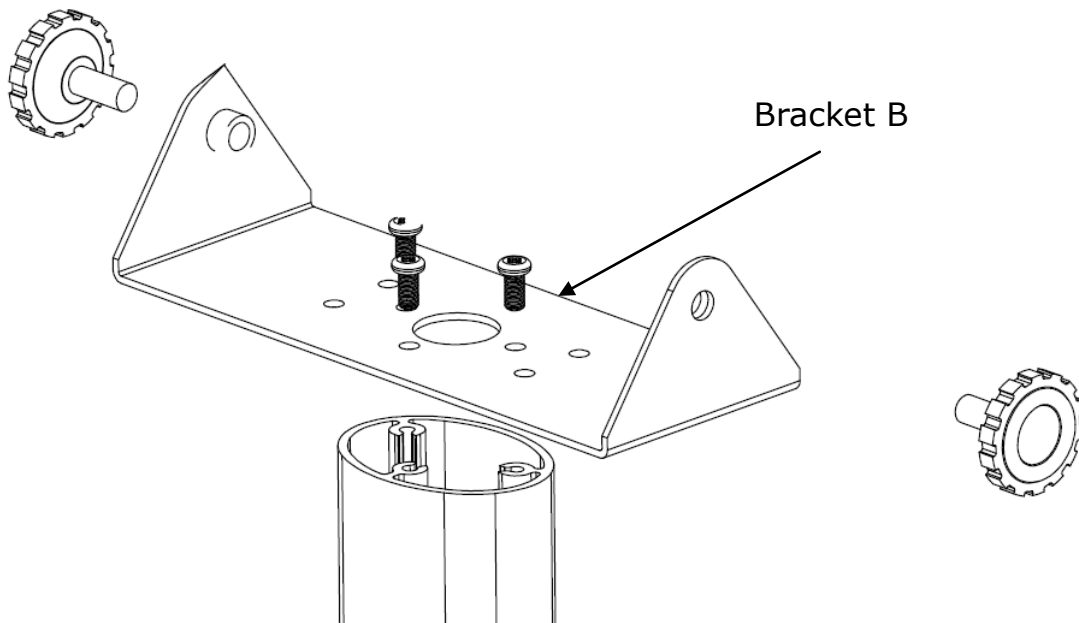


Fixing Bracket B on column with three cross screws ITEM 8.

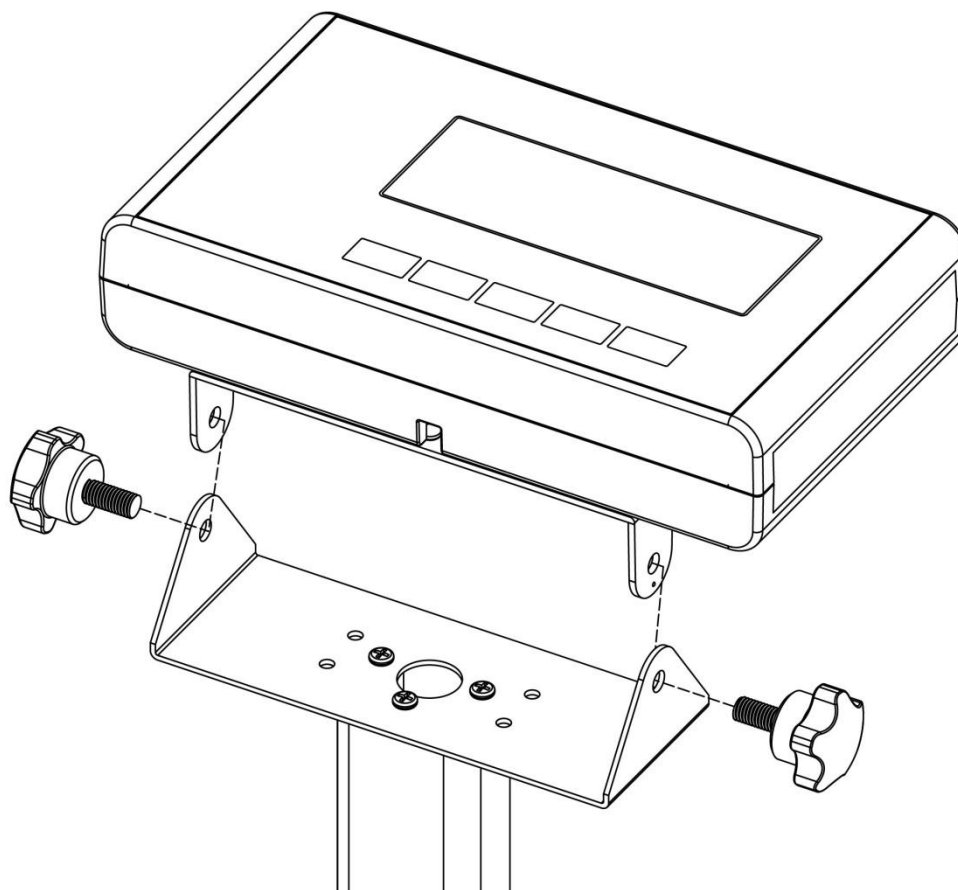


Fasten screws for indicator to complete indicator assembly.


V. Indicator Assembly-for DP37

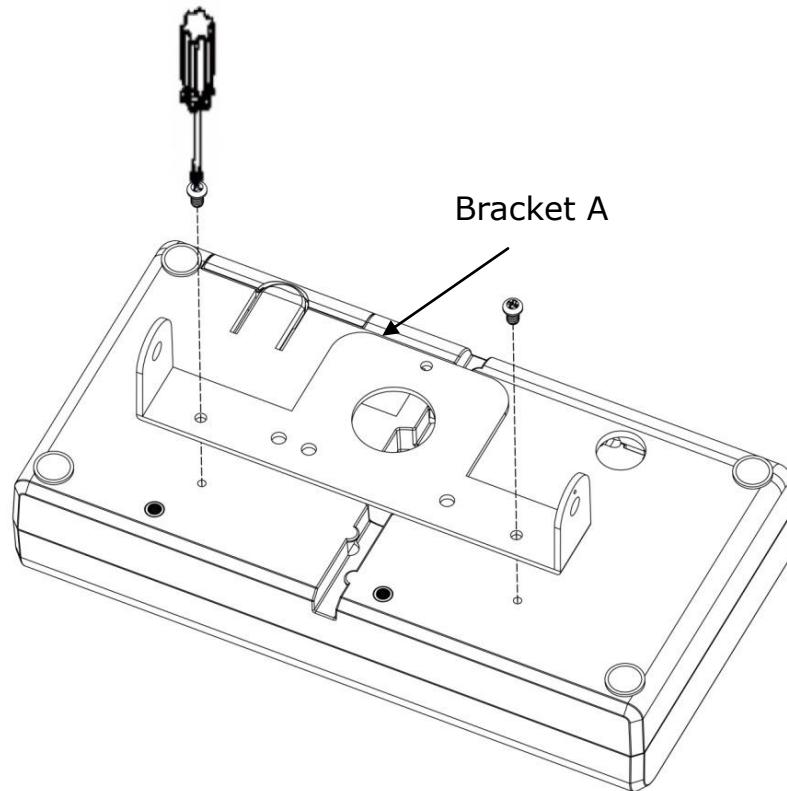

Fixing Bracket A on indicator with three screws along with indicator screw set.

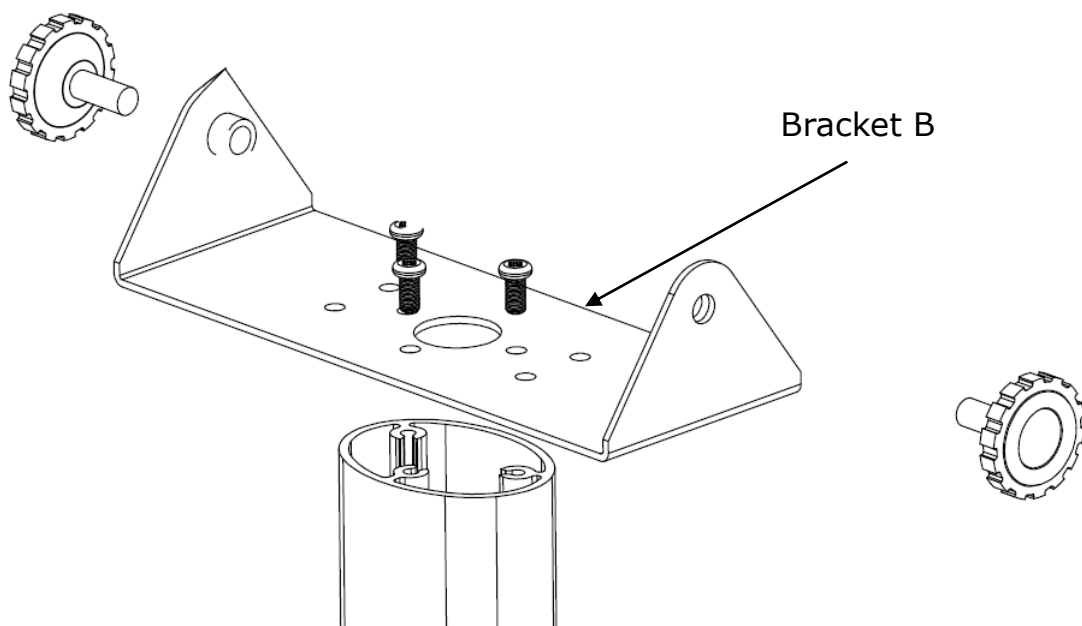


Fixing Bracket B on column with three cross screws ITEM 8.

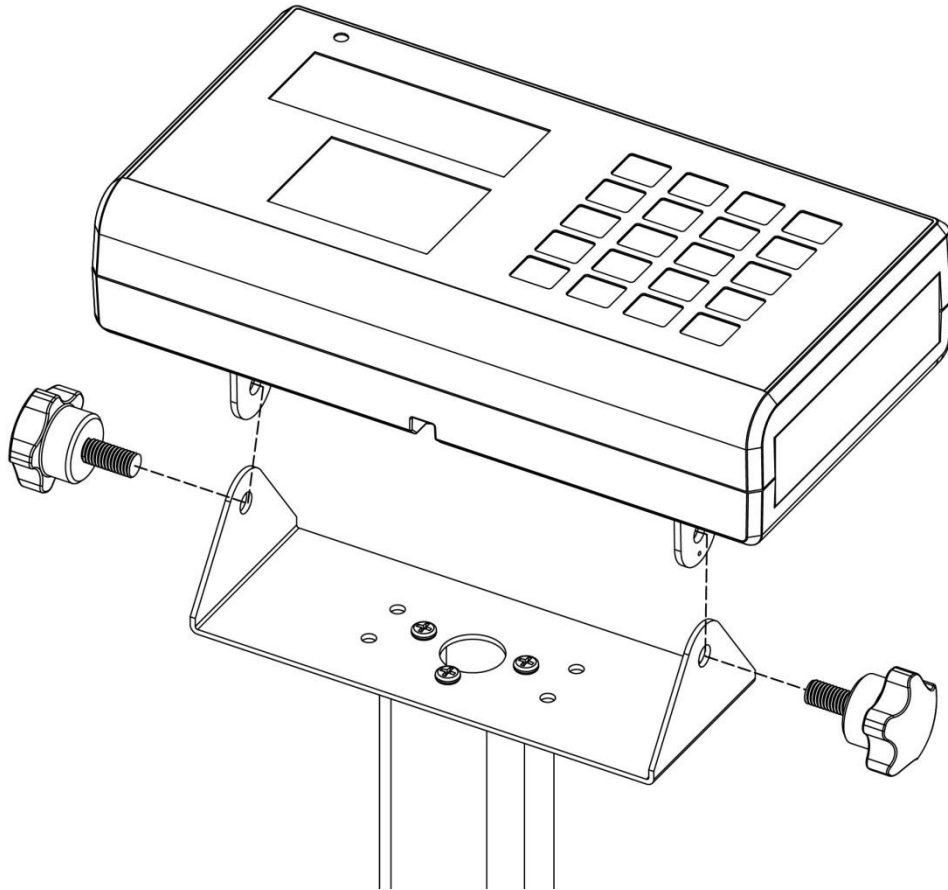


Fasten screws for indicator to complete indicator assembly.


VI. Indicator Assembly-for DP48

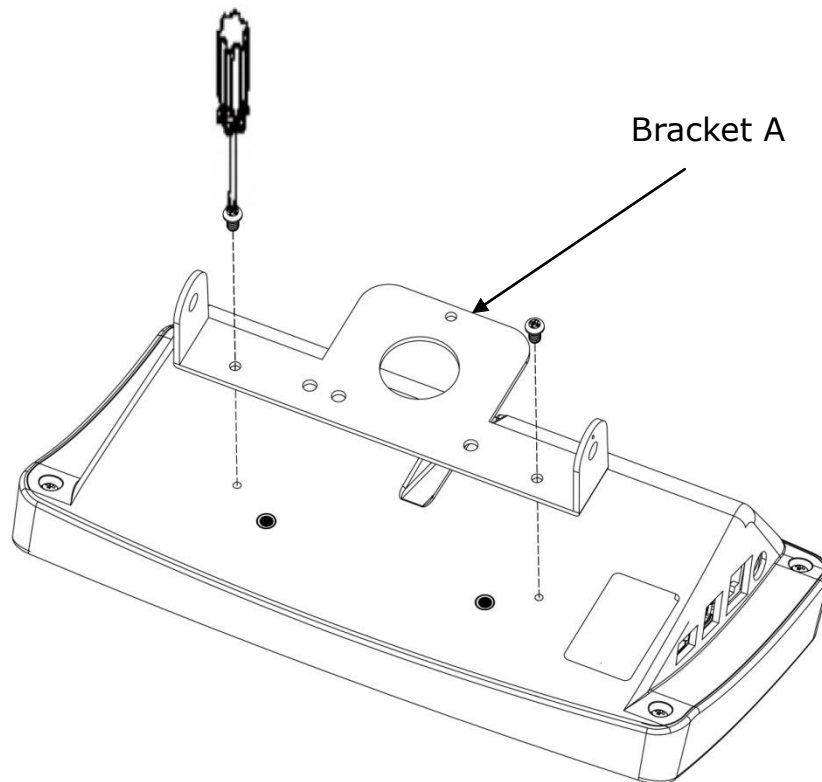

Fixing Bracket A on indicator with three screws along with indicator screw set.

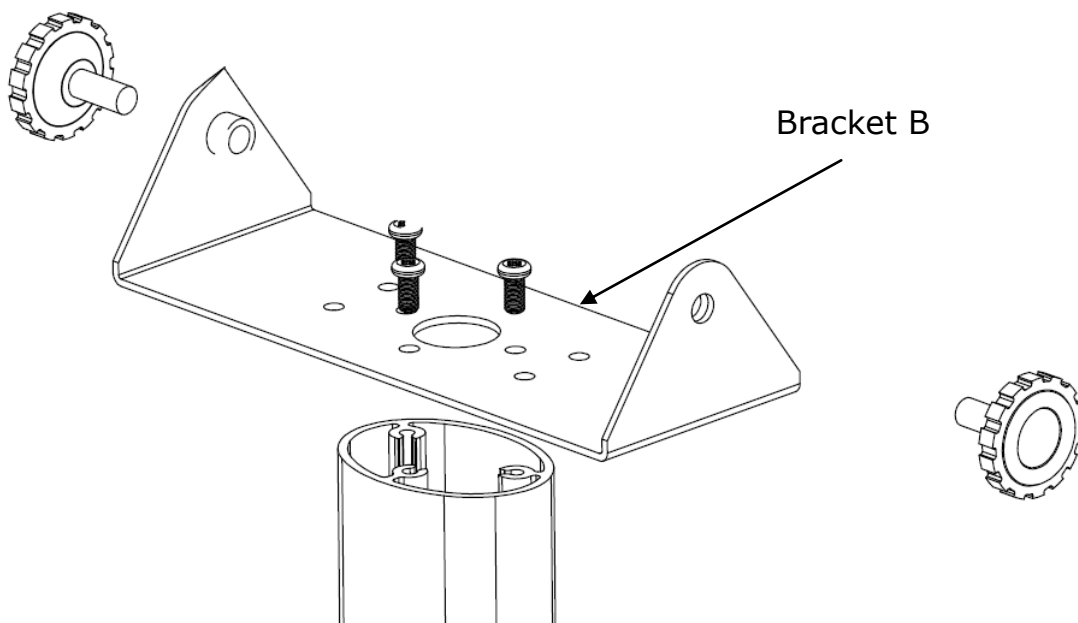


Fixing Bracket B on column with three cross screws ITEM 8.

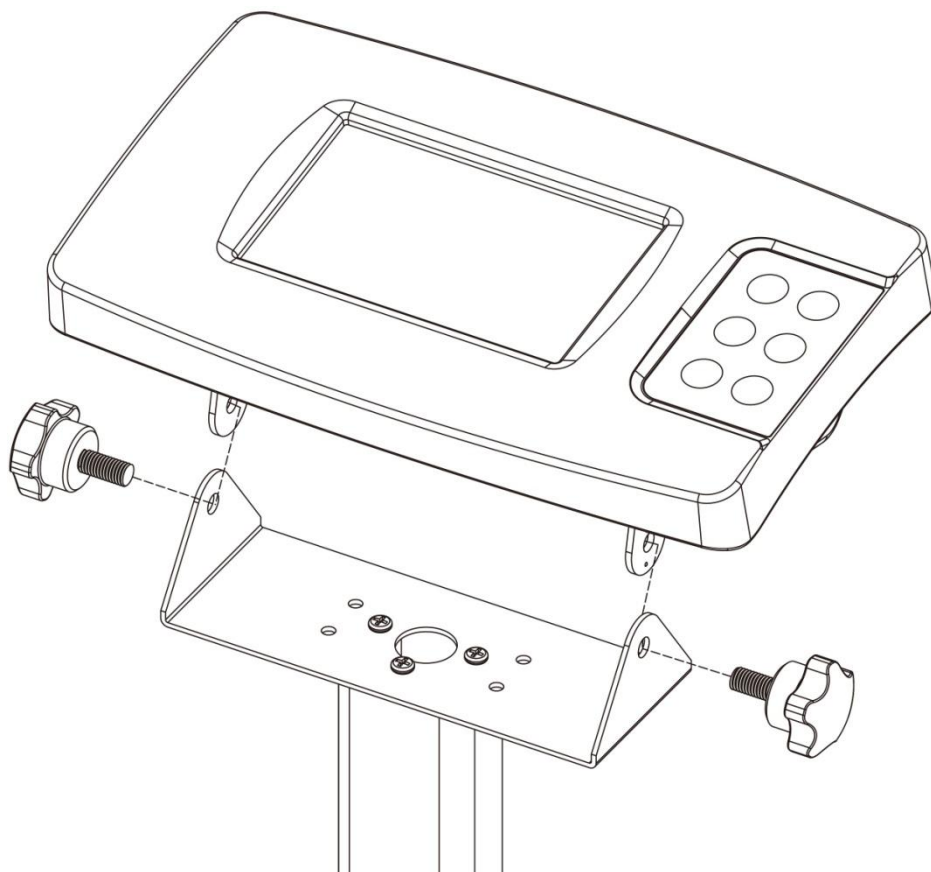

Fasten screws for indicator to complete indicator assembly.

VII. Assembly-for Thermo Printer TP21 Series

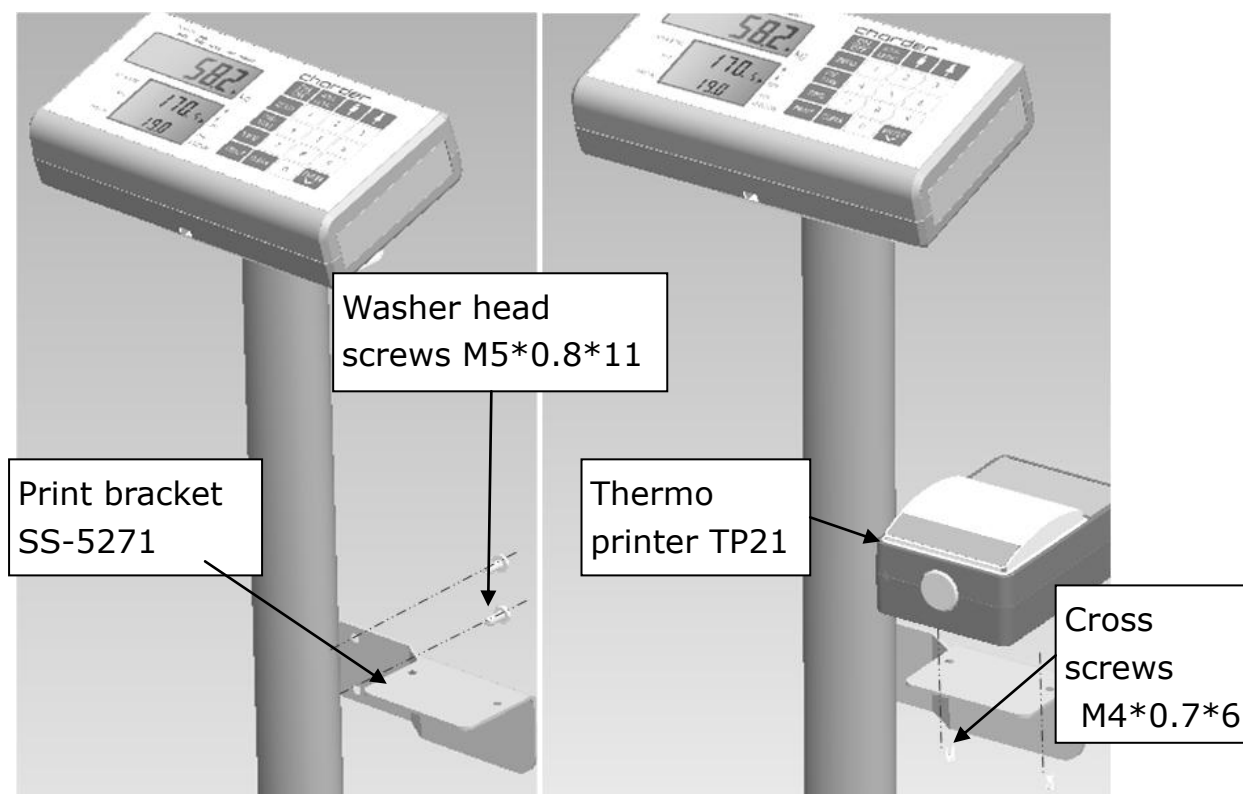

IX. Declaration of Conformity

Manufacturer hereby declares that this product is in conformity with the regulations and standards outlined in the following directives:



RoHS Directive 2011/65/EU and Delegated Directive (EU) 2015/863

Radio equipment and telecommunications terminal equipment Directive 2014/53/EU

(applicable if device has wireless function)

Part 15 of the Federal Communications Statement Rules

This device may not cause harmful interference

This device must accept interference received without causing undesired operation

(Full Declaration of Conformity is available on manufacturer's website)



Manufactured by:

Charder Electronic Co., Ltd.
No.103, Guozhong Rd., Dali Dist.,
Taichung City 41262 ,Taiwan

CD-IN-00151 REV004 02/2026