




Height Measurement

USER MANUAL **HM201M** Stadiometer

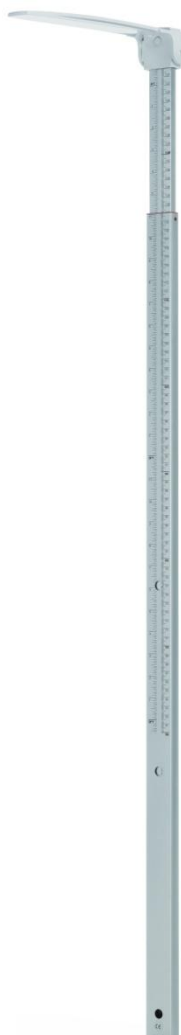

Please keep the instruction manual at hand and follow instruction for use.

CONTENTS

Explanation of Text/Symbols on Device Label/Packaging 3

Copyright Notice..... 5

I. Safety Notes 6

II. Installation..... 9

III. Using Device16

IV. Product Specifications18

V. Declaration of Conformity20

Explanation of Text/Symbols on Device Label/Packaging

Text/Symbol	Meaning
	Caution. Failure to follow instructions may result in adverse events
	Separate collection for waste of electrical and electronic equipment, in accordance with Directive 2002/96/EC. Do not dispose of device with everyday waste
	Name and address of device manufacturer
	Carefully read user manual before installation and usage, and follow instructions for use.
	Medical electrical device, Type B applied part
REF	Device catalogue number
EU REP	Name and address of authorized representative in the European Union (applicable for medical devices only)
MD	Device is a medical device. Text indicates device category type
LOT	Manufacturer's batch or lot number for device
SN	Device's serial number
#	Device's model number
UDI	Device's Unique Device Identifier (01) GTIN (11) Production date (21) Serial number
e	Value in mass units. This is the difference between two consecutive display values, used to classify and verify a scale



Device conforms to (EU) 2017/745 Regulation on Medical Devices. Four digit number is identifier for medical device Notified Body



Device claims conformity with essential regulations set by the relevant European Union legislation



Device complies with EU directives
M: Conformity label in compliance with Directive 2014/31/EU for non-automatic weighing instruments
17: Year in which conformity verification was performed and the CE label was applied. (ex: 17=2017)
0122: Identifier for metrology Notified Body



Device is a Class III scale in compliance with Directive 2014/31/EU



Name and address of entity importing device



Name and address of entity responsible for translating Information For Use

CON.

Event counter recording how many times protected-mode parameter adjustments relating to metrology-related characteristics (ex: gravity value) have been conducted for device



Device has received Taiwan NCC approval



Device conforms to U.S. Federal Communications Commission regulations



Device complies with all UK applicable product legislation



Device's polarity of power.

In case of differences, icon on device itself takes precedence.

Copyright Notice

Copyright Notice

Charder Electronic Co., Ltd.

No.103, Guozhong Rd., Dali Dist., Taichung City 41262 Taiwan

Tel: +886-4-2406 3766

Fax: +886-4-2406 5612

Website: www.chardermedical.com

E-mail: info_cec@charder.com.tw

Copyright© Charder Electronic Co., Ltd. All rights reserved.

This user manual is protected by international copyright law. All content is licensed, and usage is subject to written authorization from Charder Electronic Co., Ltd. (hereinafter Charder) Charder is not liable for any damage caused by a failure to adhere to requirements stated in this manual. Charder reserves the right to correct misprints in the manual without prior notice, and modify the exterior of the device for quality purposes without customer consent.



Charder Electronic Co., Ltd.
No. 103, Guozhong Rd., Dali Dist.,
Taichung City, 41262 Taiwan

I. Safety Notes

General Information

Thank you for choosing this Charder Medical device. It is designed to be easy and straightforward to operate, but if you encounter any problems not addressed in this manual, please contact your local Charder service partner.

Before beginning operation of the device, please read this user manual carefully, and keep it in a safe place for reference. It contains important instructions regarding installation, proper usage, and maintenance.

Intended Purpose

This medical device is designed to be used in accordance with national regulations, to measure height within specifications, for height-related usage by professionals.

Clinical Benefit

Measurement results can be used by professionals to diagnose (and monitor) height-related issues.

Intended medical indications/contraindications

Measurement: subject's body height.

Intended patient profile

- (a) Age: no restrictions
- (b) Weight: no restrictions
- (c) Patient Conditions: require measurement of body height. Can physically fit within device capacity limits and be able to stand straight (non-infant versions only).

Intended user profile

- (a) At least 20 years old
- (b) Minimum knowledge:
 - To be able to read at a high-school level and understand Arabic numerals (e.g. 1, 2, 3, 4...)
 - Basic hygiene knowledge
 - Trained in device's operation
 - Read the instruction manual
- (c) Language
 - Able to read the language of instruction manual and on-screen instructions
- (d) Qualifications
 - No special certifications or qualifications required

Residual Risk Evaluation

- (a) All foreseeable risks have been evaluated and considered acceptable. Generally speaking, the most likely risk caused by incorrect usage of the device is less accurate measurement (or inability to use device to acquire measurement), which does not pose imminent physical risk to patient or user.
- (b) Benefit-risk ratio is considered acceptable. Height measurement meters are an important option for measuring patients. Usage of device is unlikely to result in harm to user or patient.

General Handling

- Ensure all parts are properly locked and tightened before operating the device.
- Measurement accuracy requires the subject's feet, back, and head to be straightly aligned. Please note that height can vary throughout the day
- **CAUTION:** Do not use next to equipment that may cause electromagnetic or other types of interference.

Safety Instructions

Before putting device into use, please read this user manual carefully. It contains important instructions for installation, usage, and maintenance of device.

The manufacturer shall not be liable for damages caused by failure to heed the following instructions:

- The device has an expected service life of 5 years when correctly handled, serviced, and periodically inspected in accordance with manufacturer's instructions.
- Improper installation will render the warranty null and void.
- Observe permissible ambient temperatures for use

Cleaning

Device surface should be cleaned using alcohol-based wipes.

Maintenance

Please contact your local Charder distributor for regular maintenance and calibration, regular checking of accuracy is recommended; frequency to be determined by level of use and state of device.

Warranty/Liability


- The period of warranty shall be eighteen(18) months, beginning on the date of purchase. Please retain your receipt as proof of purchase.
- No responsibility shall be accepted for damage caused through any of the following reasons: unsuitable or improper storage or use, incorrect installation or commissioning by the owner or third parties, natural wear and tear, changes or modifications, incorrect or negligent handling, chemical, electrochemical, or electrical interference, unless damage is attributable to negligence on the part of Charder.
- This device does not contain any user-maintained parts. All maintenance, technical inspections, and repairs should be conducted by an authorized Charder service partner, using original Charder accessories and spare parts. Charder is not liable for any damages arising from improper maintenance or usage. Dismantlement of the device will void the warranty.

Incident Reporting


Any serious incident that has occurred in relation to the device should be reported to the manufacturer, EU representative (if device is used in EU member state), and competent authority of user/subject's member state.

II. Installation


1. Make a mark 54 cm off the ground.

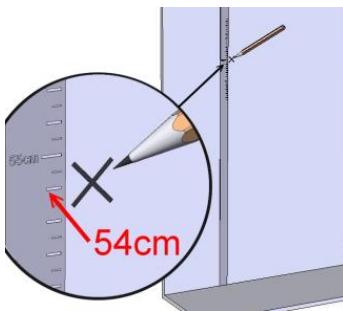


2. Align the mark with one hole, and use it and bracket to mark position for second hole.

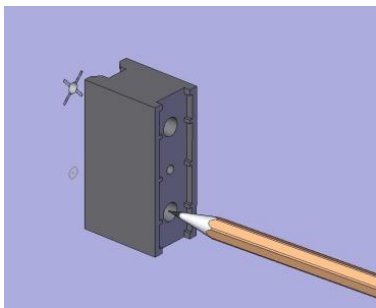


3. Drill through both markings.

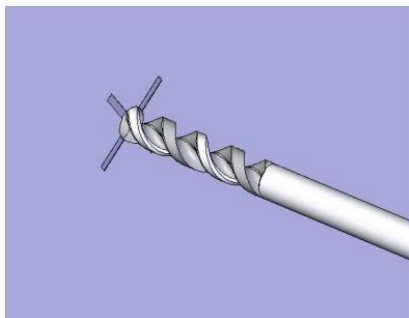


4. Insert plastic nails firmly.

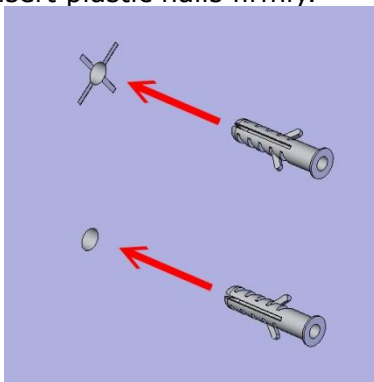

5. Screw bracket to wall

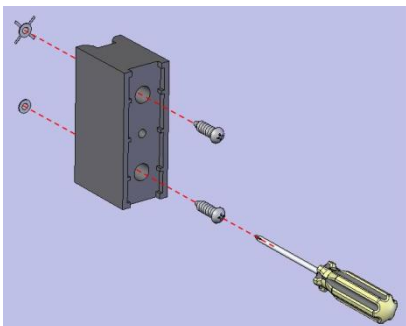


6. Screw height rod to bracket

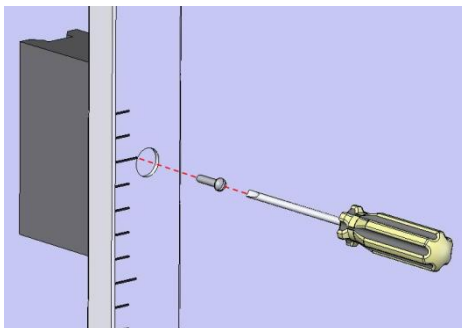


Without Bracket


1. Make a mark 51.5 cm off the ground.

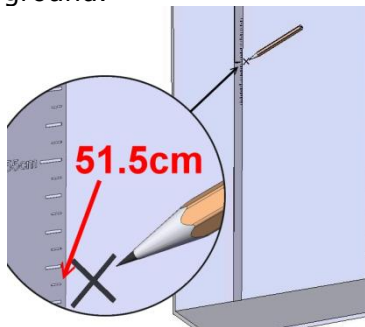

2. Extend the height rod fully, and mark the top hole position

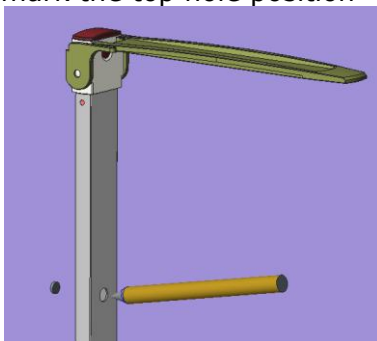


3. Drill through both markings to secure HM201M to wall..

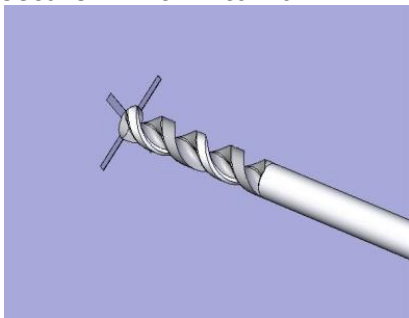


Attaching to compatible Charder scales

MS39-series

1. Attach bracket

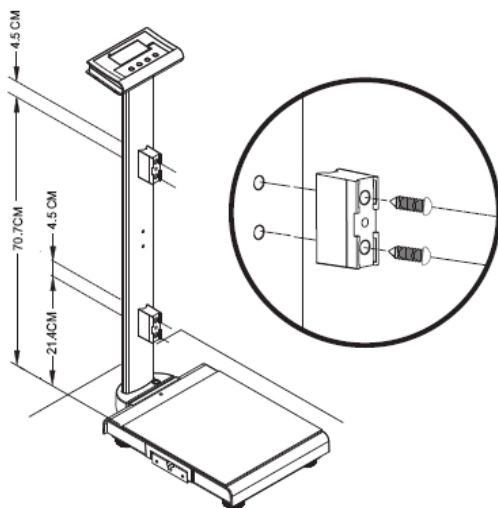


2. Screw height rod to bracket

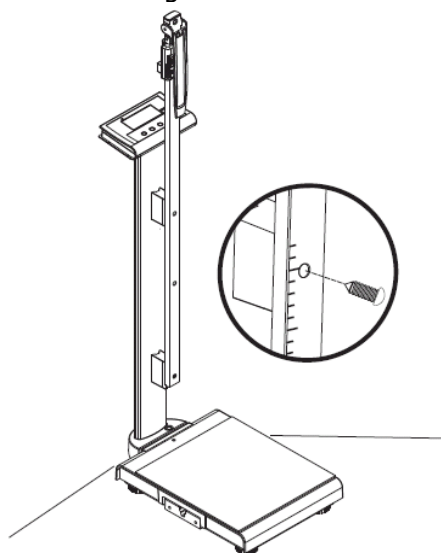


MS49-series

1. Attach fixing block using cross head screws

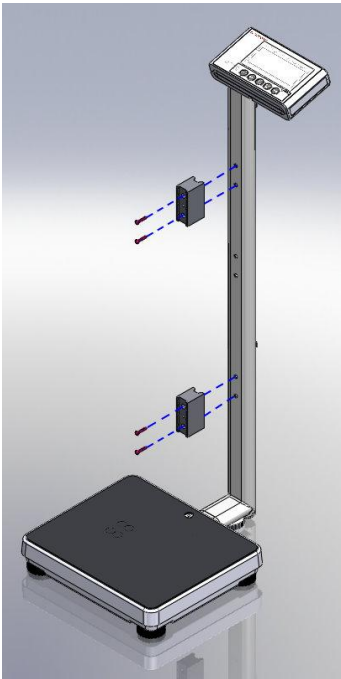

2. Attach HM201M to fixing block using flat head screws

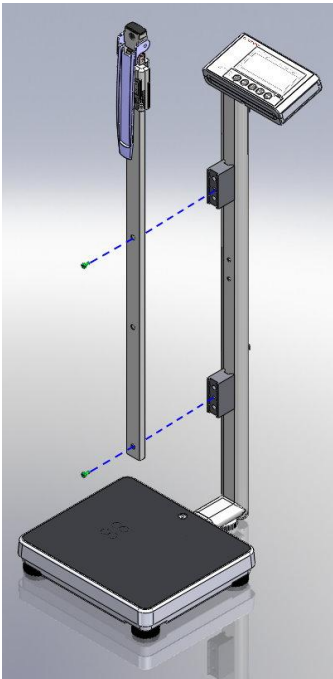


Parts


		
Fixing block*2	Flat head screw*2	Cross head screw*2

Assemble HM201M on the scale with wheel castors

HM201M standard accessories for scale with castors

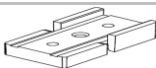No.	Accessories	Item	Spec.	Qty.
1		Side Bracket for height rod	SS-5721	2
2		Fixing block for height rod	ST-3331	2
3		Screw	M5*11	8
4		Cable clip	CHC-3	2
5		Flat head screw	M5*10	2


1. Fix two brackets on the column

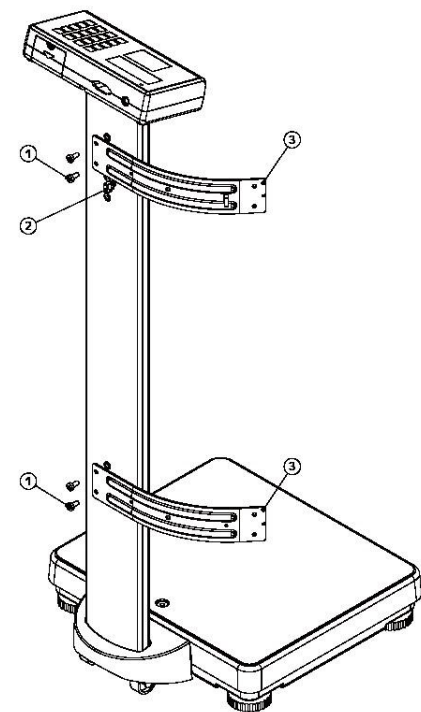

Item	Name	Quantity
1	M5x11L screw	4
2	Cable clip	2
3	Bracket for height rod	2

*Photo of display for reference only.
Please refer to the actual product.


2. Fix HM201M on the bracket and tight the screw

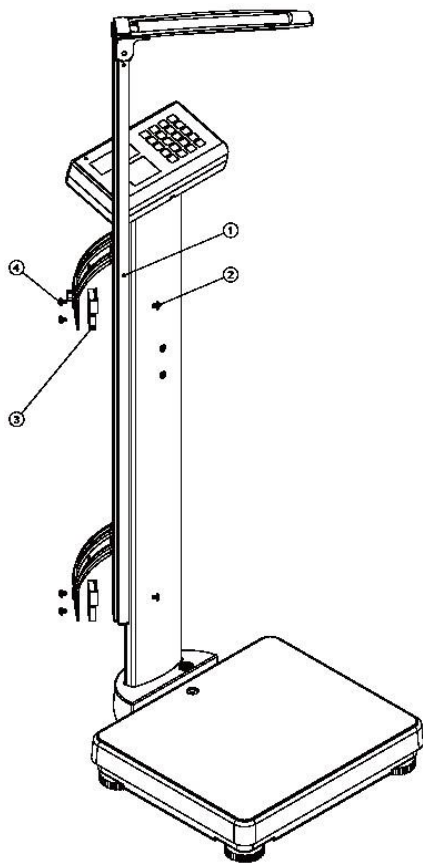


Item	Name	Quantity
1	HM200D/ 201D/ 201M	1
2	M5x10L Flat head screw	2
3	Fixing block for height rod	2
4	M5x11L screw	4

III. Using Device


Over 121cm

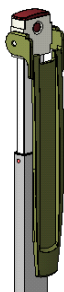

1. Extend rod

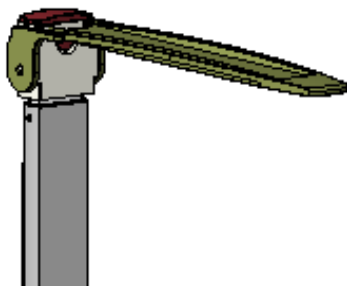


2. Raise headpiece to horizontal position

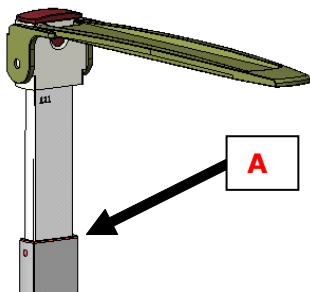


3. Head reading should be taken at point A


Under 120 cm

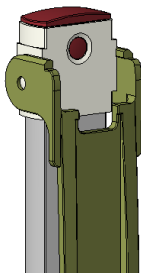

1. Fold down headpiece and press on buckle

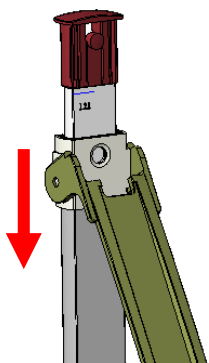

2. While pressing buckle, press down headpiece

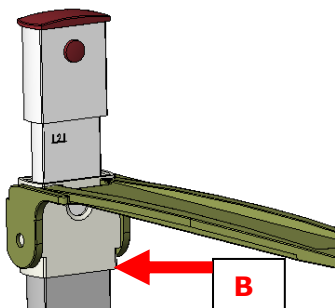

3. Height reading should be taken at point B

IV. Product Specifications

A. Device Information

Model		HM201M
Height Measurement	Range	60-210 cm 24-82 3/4"
		Side bracket 80-210cm 31 1/2-24-82 3/4"
	Graduation	1 mm 1/16 in
	Accuracy	±5 mm
Dimensions	Overall	40(W) x 30(D) x 1780(H) mm
	Device Weight	About 0.7 kg
Operation Environment		+5°C ~ +35°C 15%RH ~ 85%RH 700 hPa ~1060 hPa

[illegible]

V. Declaration of Conformity

Manufacturer hereby declares that this product is in conformity with the regulations and standards outlined in the following directives:

	(EU) 2017/745 Regulation on Medical Devices
---	--

RoHS Directive 2011/65/EU and Delegated Directive (EU) 2015/863

(applicable if device is electronic device)

Radio equipment and telecommunications terminal equipment Directive 2014/53/EU

(applicable if device has wireless function)

Part 15 of the Federal Communications Statement Rules

This device may not cause harmful interference

This device must accept interference received without causing undesired operation

(Full Declaration of Conformity is available on manufacturer’s website)

Authorized EU Representative:



Obelis s.a.

Bd Général Wahis, 53
B-1030 Brussels
Belgium

Manufactured by:



Charder Electronic Co., Ltd.
No.103, Guozhong Rd., Dali Dist.,
Taichung City 41262 ,Taiwan

CD-IN-1025 7026X 02/2026