


Manually installing USB drivers


(MA60I/MA80I connected to Charder Insight) (Windows)

- 1 **Scenario:** Charder Insight is unable to connect with MA60I/MA80I Body Composition Analyzer, even though UA-010 USB Transfer Key Cable is already plugged in. USB driver needs to be installed manually on PC.
- 2 **Applicable operating systems:**
Windows 10, Windows 11
- 3 **Checking if drivers need to be installed:**
 - Plug UA-010 Cable into Windows PC
 - Press [Windows Key] + [R] to open the “Run Command”. Type “devmgmt.msc” and press [Enter].
 - Under “Other devices”, confirm if an exclamation mark “!” is seen next to “USB Serial Port”. If the exclamation mark appears, then USB drivers need to be installed.

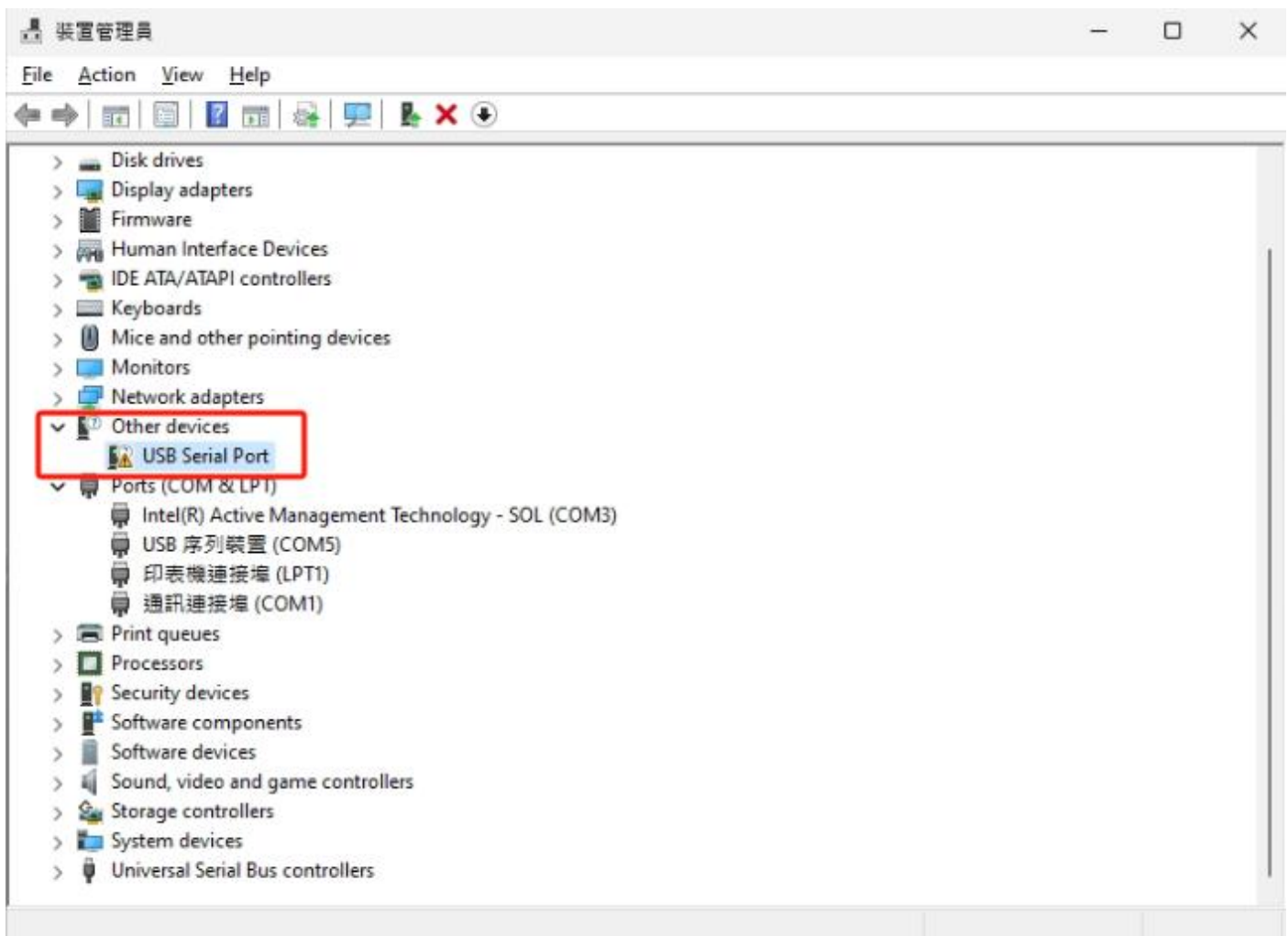


4 Installing USB driver:


- After downloading CDM212364_Setup.zip file from [Charder Medical website](#), extract “CDM212364_Setup.exe” and run installation program.

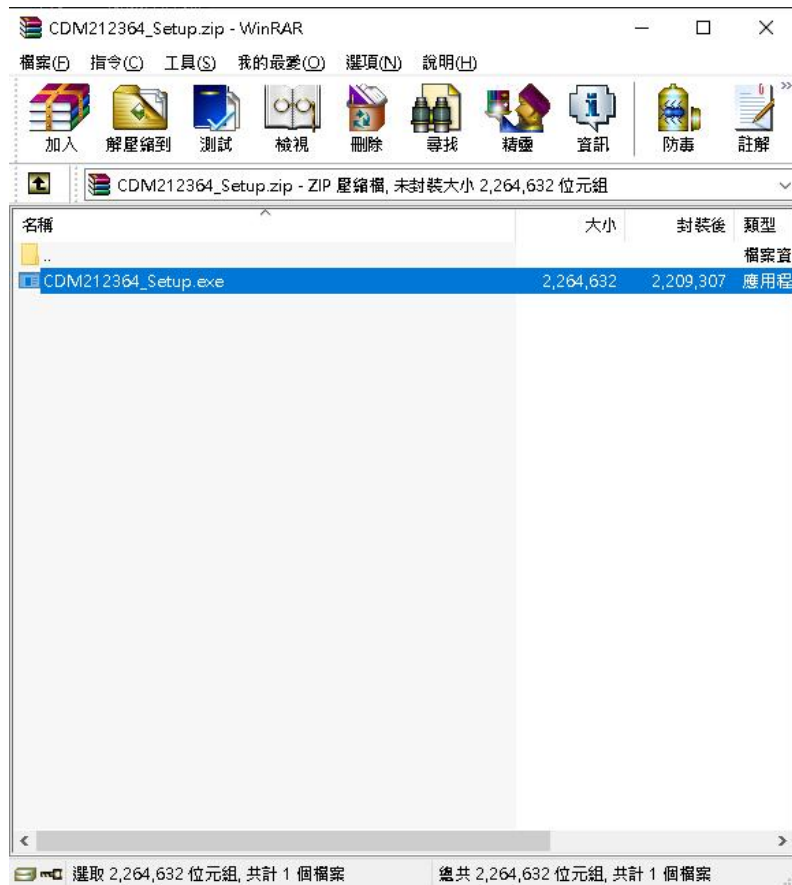

- Follow the installation instructions:.



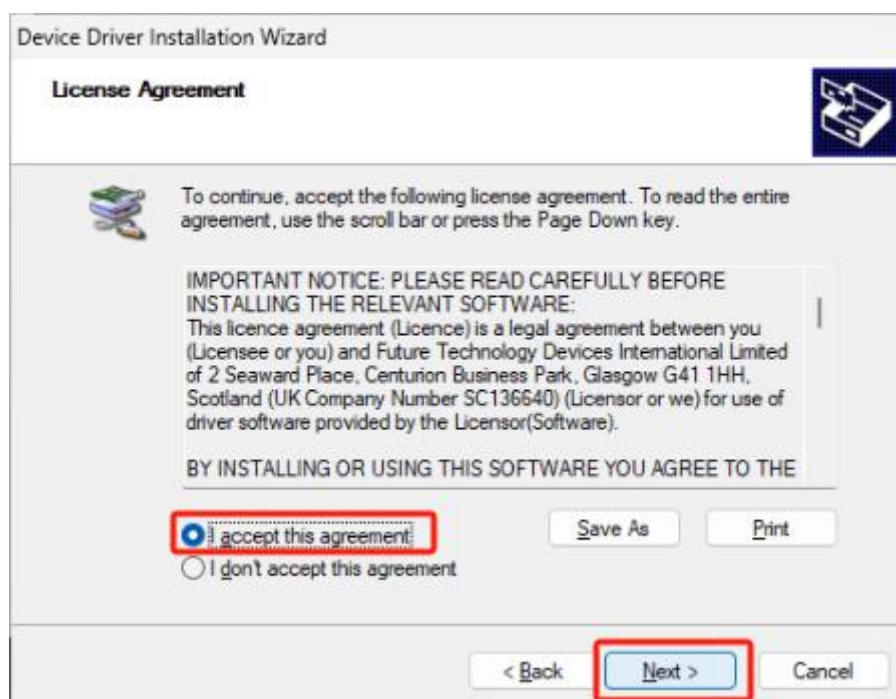

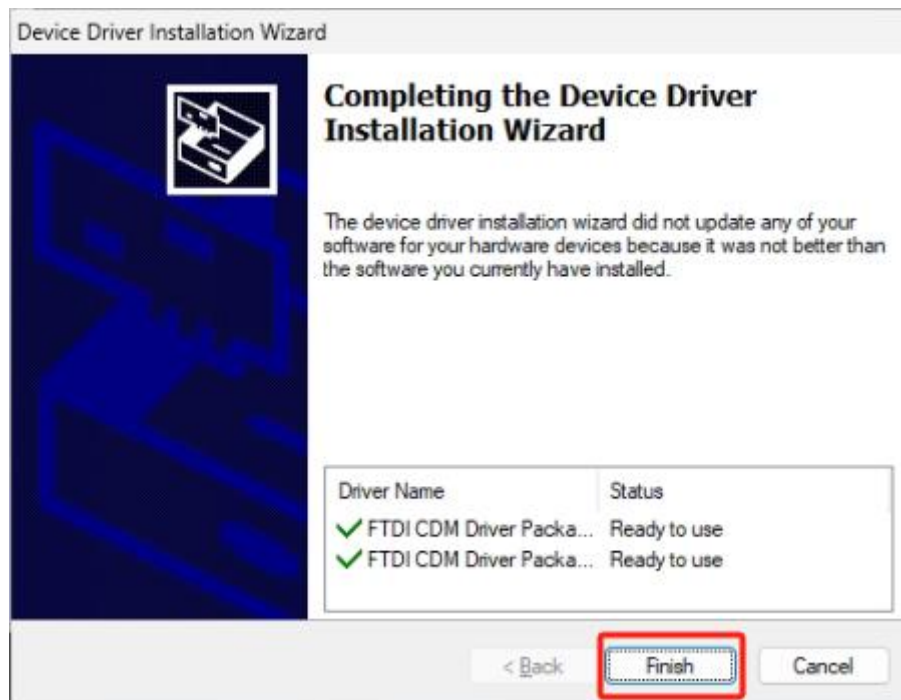

- After driver is installed, exclamation mark “!” will disappear. Confirm if MA60I/MA80I can connect to Charder Insight now.

