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MS5980

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| Guidance and manufacturer's declaration-electromagnetic emissions | | |
|---|------------|---|
| The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment. | | |
| Emission test | Compliance | Electromagnetic environment-guidance |
| RF emissions CISPR 11 | Group 1 | The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. |
| RF emissions CISPR 11 | Class A | The product is suitable for use in all establishments other than domestic and those directly connected to a low voltage power supply network which supplies buildings used for domestic purposes. |

| Guidance and manufacturer's declaration-electromagnetic immunity | | | |
|---|---|---|--|
| The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment. | | | |
| Immunity test | IEC 60601 test level | Compliance level | Electromagnetic environment-guidance |
| Electrostatic discharge(ESD) IEC 61000-4-2 | <u>± 8 kV contact</u> <u>± 2 kV, ± 4 kV,</u> <u>± 8 kV, ± 15 kV</u> <u>air</u> | <u>± 8 kV contact</u> <u>± 2 kV, ± 4 kV,</u> <u>± 8 kV, ± 15 kV</u> <u>air</u> | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30% |
| Power frequency(50/60 Hz) magnetic field IEC 61000-4-8 | <u>30 A/m</u> | <u>30 A/m</u> | The device power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. |
| NOTE UT is the a.c. mains voltage prior to application of the test level. | | | |

Guidance and manufacturer's declaration-electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that is used in such an environment.

| Immunity test | IEC 60601 test level | Compliance level | Electromagnetic environment-guidance |
|---------------------------|---------------------------|----------------------------------|---|
| Radiated RF IEC 61000-4-3 | 3 V/m 80MHz to 2,7 GHz | 3 V/m <u>80MHz to 2,7 GHz</u> | <p>Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800MHz to 2,5 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p>  |

NOTE1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the device.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distance between portable and mobile RF communications equipment and the Device

The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter

W

Separation distance according to frequency of transmitter m

150 kHz to 80 MHz

$d = 1,2\sqrt{P}$

80 MHz to 800 MHz

$d = 1,2\sqrt{P}$

800 MHz to 2,7 GHz

$d = 2,3\sqrt{P}$

0,01

0,12

0,12

0,23

0,1

0,38

0,38

0,73

1

1,2

1,2

2,3

10

3,8

3,8

7,3

100

12

12

23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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The top screenshot shows the 'Pre-tare' screen with a 'Back' button at the top left. The display shows '0-.-.-' followed by 'kg'. Below the display is a 'CLEAR' button. To the right is a numeric keypad with digits 1-9, 0, and an 'Enter' button. The bottom screenshot shows the same 'Pre-tare' screen, but the display now shows '5.000' followed by 'kg'. The 'CLEAR' button is still present, and the numeric keypad remains on the right.

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The screenshot shows the main interface of the scale. At the top, there are fields for 'Patient:' and 'Nurse:' with close buttons. Below these are status icons for information, Wi-Fi, and battery. A section labeled 'Awaiting transfer:' contains a 'PRE-TARE' button (highlighted with a red box) and a 'SCAN' button. The main display shows a large '0' followed by a minus sign (highlighted with a red box) and '5.000kg'. Below the display are fields for 'Height (cm)' and 'BMI'. At the bottom, a status bar shows 'Max 20 kg', 'Min 0.1 kg', and 'e=0.005 kg'.

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

| 敘述 | 顯示器 |
|--|---|
| <p>按 [MEMORY] 鍵能切換顯示器已存的預扣重值 (編號 01-10)</p> <p>注意：新增預扣重值請長按[MEMORY] 鍵 3 秒進入設定 (請看上一段說明)</p> |  <p>The screenshot shows the scale's main display. At the top, there are fields for 'Patient:' (red background) and 'Nurse:' (yellow background). Below these are 'Awaiting transfer:' and 'PRE-TARE' buttons. A red box highlights the number '3' next to the 'PRE-TARE' button. The large central display shows '-0.500kg'. Below this are input fields for 'Height (cm)' and 'BMI'. At the bottom, it shows 'Max 20 kg', 'Min 0.1 kg', and 'e=0.005 kg'.</p> |

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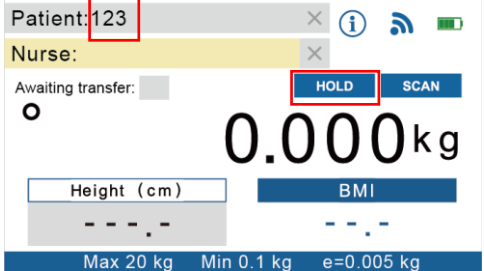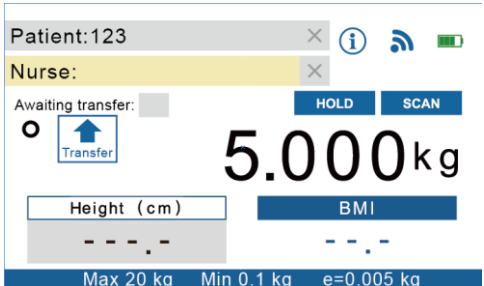
| 敘述 | 顯示器 |
|--|--|
| <p>輸入嬰兒的 ID，並按 [Hold] 鍵啟動 Hold 功能</p> |  <p>The screenshot shows the scale's main display. The 'Patient:' field now contains the number '123'. The 'Nurse:' field is empty. The 'Awaiting transfer:' button is active. A red box highlights the 'HOLD' button. The large central display shows '0.000kg'. Below this are input fields for 'Height (cm)' and 'BMI'. At the bottom, it shows 'Max 20 kg', 'Min 0.1 kg', and 'e=0.005 kg'.</p> |
| <p>將嬰兒放置於秤台上，待重量鎖定 (例：嬰兒授乳前重量 5.000kg)</p> |  <p>The screenshot shows the scale's main display. The 'Patient:' field contains '123'. The 'Nurse:' field is empty. The 'Awaiting transfer:' button is active. A red box highlights the 'Transfer' button (with an upward arrow icon). The large central display shows '5.000kg'. Below this are input fields for 'Height (cm)' and 'BMI'. At the bottom, it shows 'Max 20 kg', 'Min 0.1 kg', and 'e=0.005 kg'.</p> |

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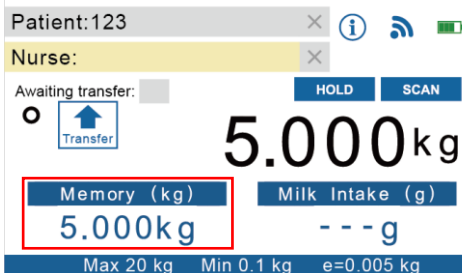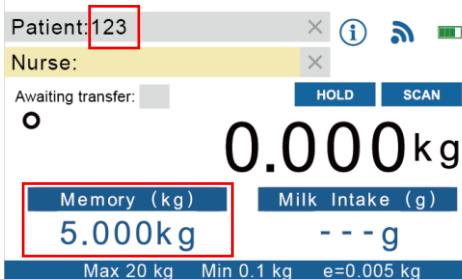
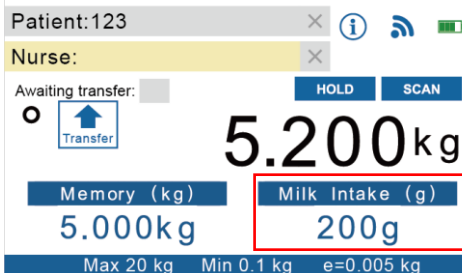
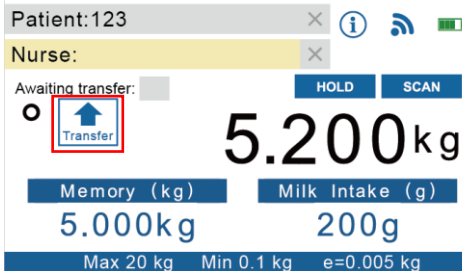
| EXIT | | General | |
|---------------|-------|----------------|-------|
| Auto Off Time | 180s | G-Compensation | ----- |
| Backlight | High | URL Host | ----- |
| Buzzer | On | Auto Hold | On |
| Data Transfer | On | Auto Transfer | On |
| Date/Time | ----- | | |
| Wifi Setting | ----- | | |

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Copyright© Charder Electronic Co., Ltd. 版權所有。