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Guidance and manufacturer's declaration-electromagnetic emissions		
The product isintended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The product uses RFenergy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronequipment.
RF emissions CISPR 11	Class A	The product is suitable for use in all establishments other than domesticand those directly connected to a low voltage power supply network which supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Compliance	

Guidance and manufacturer's declaration-electromagnetic immunity

The product is intended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge(ESD) IEC 61000-4-2	<u>±8 kV contact</u> <u>±2 kV, ±4 kV, ±8 kV, ±15 kV air</u>	<u>±8 kV contact</u> <u>±2 kV, ±4 kV, ±8 kV, ±15 kV air</u>	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/burst IEC 61000-4-4	<u>+ 2kV for power supply lines</u>	<u>+ 2kV for power supply lines</u>	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	<u>+ 1kV line(s) to line(s)</u> <u>+ 2kV line(s) to earth</u>	<u>+ 1kV line(s) to line(s)</u> <u>+ 2kV line(s) to earth</u>	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<u>0% UT for 0.5 cycle</u> <u>0% UT for 1 cycle</u> <u>70% UT(30% dip in UT) for 25cycles</u> <u>0% UT for 5 s</u>	<u>0% UT for 0.5 cycle</u> <u>0% UT for 1 cycle</u> <u>70% UT(30% dip in UT) for 25cycles</u> <u>0% UT for 5 s</u>	Mains power quality should be that of a typical commercial or hospital environment. If the user of the product requires continued operation during power mains interruptions, it is recommended that the product be powered from an uninterruptible power supply or a battery.
Power frequency(50, 60 Hz) magnetic field IEC 61000-4-8	<u>30 A/m</u>	30 A/m	The product power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE UT is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration-electromagnetic immunity

The product is intended for use in the electromagnetic environment specified below.

The customer or the user of the product should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 KHz to 80 MHz <u>6 V in ISM bands between 0.15 MHz and 80 MHz</u> <u>80 % AM at 1 kHz</u>	3 Vrms 150 KHz to 80 MHz <u>6 V in ISM bands between 0.15 MHz and 80 MHz</u> <u>80 % AM at 1 kHz</u>	Portable and mobile RF communications equipment should be used no closer to any part of the product including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800MHz to 2,7GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m <u>80MHz to 2,7 GHz</u>	3 V/m <u>80MHz to 2,7 GHz</u>	

NOTE1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE2

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the product is used exceeds the applicable RF compliance level above, the product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the product.
- b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

**Recommended separation distance between
portable and mobile RF communications equipment and the product**

The product is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the product can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the product as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,7 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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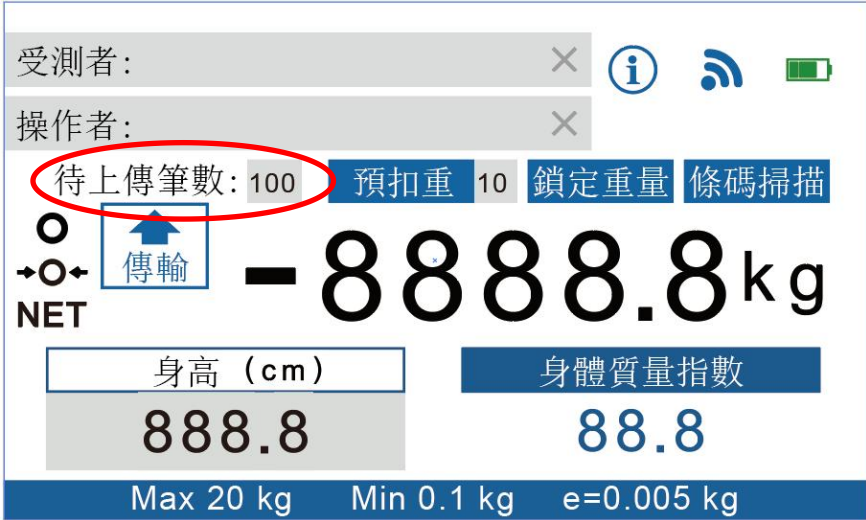
The image shows a digital scale's display interface. At the top, there are fields for '受測者:' (Subject) and '操作者:' (Operator), both with 'X' icons to clear them. To the right are icons for information (i), Wi-Fi, and battery status. Below these, '待上傳筆數: 100' (Number of items to be uploaded: 100) is circled in red. Next to it are buttons for '預扣重 10' (Pre-weigh 10), '鎖定重量' (Lock weight), and '條碼掃描' (Barcode scan). The main display shows a large weight of '- 88888.8 kg'. To the left of the weight is a 'NET' label and a '傳輸' (Transfer) button with an upward arrow icon. Below the weight, there are two boxes: '身高 (cm)' (Height) showing '888.8' and '身體質量指數' (BMI) showing '88.8'. At the bottom, a blue bar displays 'Max 20 kg', 'Min 0.1 kg', and 'e=0.005 kg'.

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

The screenshot shows a digital scale interface. At the top, there are fields for '受測者:' (Subject) and '操作者:' (Operator), each with a close button (X) and an information icon (i). Below these are '待上傳筆數:' (Number of items to upload) and a '預扣重' (Tare weight) button, which is highlighted with a red box. To the right of the tare weight button is a '條碼掃描' (Barcode scan) button. The main display shows a large '5.000 kg' with a minus sign in a red box to its left, indicating a tare weight. Below the main display are two input fields: '身高 (cm)' (Height in cm) and '身體質量指數' (Body Mass Index). At the bottom, there is a status bar with 'Max 20 kg', 'Min 0.1 kg', and 'e=0.005 kg'.

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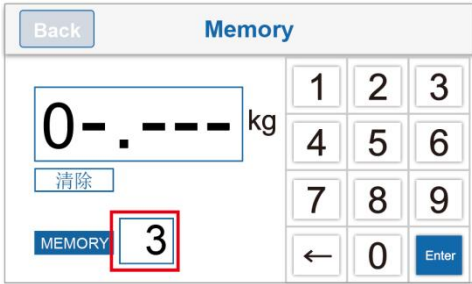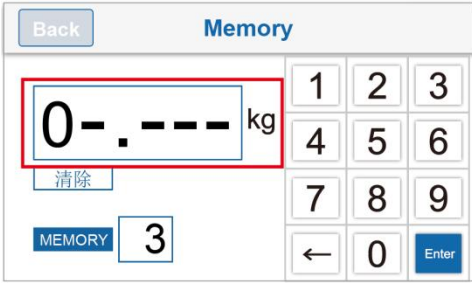[illegible]

Notes

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

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  0122	(EU) 2017/745 Regulation on Medical Devices
  8506	2014/31/EU Non-automatic Weighing Instruments Directive (OIML models only)

RoHS Directive 2011/65/EU and Delegated Directive (EU) 2015/863

Radio equipment and telecommunications terminal equipment Directive 2014/53/EU
(applicable if wireless module is used)

Part 15 of the Federal Communications Statement Rules
This device may not cause harmful interference.
This device must accept any interference received, including interference that may cause undesired operation

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Copyright© Charder Electronic Co., Ltd. 版權所有。