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Fax: +886-4-2406 5612

Website: www.chardermedical.com E-mail: info_cec@charder.com.tw

Copyright© Charder Electronic Co., Ltd. All rights reserved.

This user manual is protected by international copyright law. All content is licensed, and usage is subject to written authorization from Charder Electronic Co., Ltd. (hereinafter Charder) Charder is not liable for any damage caused by a failure to adhere to requirements stated in this manual. Charder reserves the right to correct misprints in the manual without prior notice, and modify the exterior of the device for quality purposes without customer consent.

Charder Electronic Co., Ltd.
No. 103, Guozhong Rd., Dali Dist.,
Taichung City, 41262 Taiwan

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Guidance and manufacturer' s declaration-electromagnetic emissions		
The MS5980 Infant Scale is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Compliance	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

Guidance and manufacturer' s declaration-electromagnetic immunity			
The MS5980 Infant Scale is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge(ESD) IEC 61000-4-2	<u>± 8 kV contact</u> <u>± 2 kV, ± 4 kV,</u> <u>± 8 kV, ± 15 kV</u> <u>air</u>	<u>± 8 kV contact</u> <u>± 2 kV, ± 4 kV, ±</u> <u>8 kV, ± 15 kV air</u>	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%

Electrical fast transient/burst IEC 61000-4-4	± 2kV for power supply lines + 1kV for input/output lines	+ 2kV for power supply lines + 1kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1kV line(s) to line(s) ± 2kV line(s) to earth	+ 1kV line(s) to line(s) + 2kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<u>0% UT for 0,5 cycle</u> <u>0% UT for 1 cycle</u> <u>70% UT(30% dip in UT) for 25 cycles</u> <u>0% UT for 5 s</u>	<u>0% UT for 0,5 cycle</u> <u>0% UT for 1 cycle</u> <u>70% UT(30% dip in UT) for 25 cycles</u> <u>0% UT for 5 s</u>	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency(50/60 Hz) magnetic field IEC 61000-4-8	<u>30 A/m</u>	<u>30 A/m</u>	The device power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

Guidance and manufacturer' s declaration-electromagnetic immunity			
The MS5980 Infant Scale is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 KHz to 80 MHz	3 Vrms 150 KHz to 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of the device including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	<u>6 V in ISM bands between 0,15 MHz and 80 MHz</u> <u>80 % AM at 1 kHz</u>	<u>6 V in ISM bands between 0,15 MHz and</u>	

	3 V/m 80MHz to 2,7 GHz	<u>80 MHz</u> <u>80 % AM at 1</u> <u>kHz</u> 3 V/m <u>80MHz to 2,7</u> <u>GHz</u>	Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b. Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the device. b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended separation distance between portable and mobile RF communications equipment and the MS5980 Infant Scale			
The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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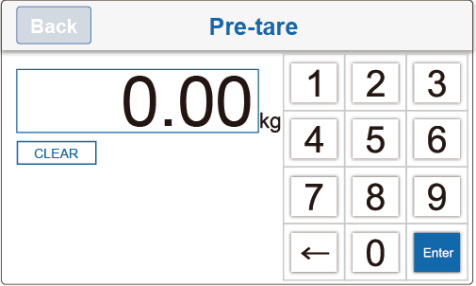
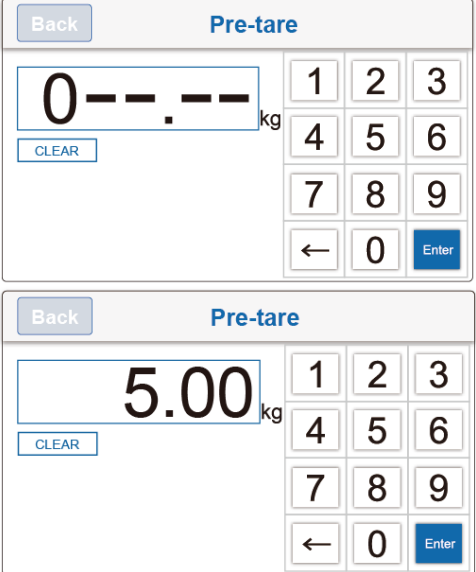
敘述	顯示器
按 [PRE-TARE] 鍵。從最左側的數字開始，輸入預扣重量值	 The image shows a digital scale display with a light blue background. At the top left is a 'Back' button. In the center, the text 'Pre-tare' is displayed in blue. Below it, the weight '0.00' is shown in large black digits, followed by a small 'kg' unit. To the left of the digits is a 'CLEAR' button. To the right is a numeric keypad with buttons for digits 1-9, 0, and a left arrow. An 'Enter' button is at the bottom right of the keypad.
用數字鍵 0-9 輸入預扣重值 例：預扣值 5.0 kg，請按 0-0-0-5-0 例：預扣值 13.5 kg，請按 0-0-1-3-5 按 [Enter] 鍵確認預扣重值	 The image contains two sequential screenshots of the scale display. The top screenshot shows the display with '0--.--' and a decimal point, indicating it's ready for input. The bottom screenshot shows the display with '5.00' and the 'kg' unit, indicating the input has been accepted. Both screenshots include the same 'Back', 'Pre-tare', 'CLEAR', and numeric keypad layout as the first image.

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Patient: [X] [i] [Wi-Fi] [Battery]

Nurse: [X]

Awaiting transfer: 0 [PRE-TARE] [HOLD] [SCAN]

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The screenshot shows a medical scale interface. At the top, there are fields for 'Patient:' and 'Nurse:', each with a close button (X). Below these, it says 'Awaiting transfer: 0' followed by a blue button labeled 'PRE-TARE' and a small box containing the number '3'. The main display shows a large weight of '-13.5 kg'. Below the weight, there are fields for 'Height' (---.- cm) and 'BMI' (---.-). At the bottom, it shows 'Max 300 kg', 'Min 2 kg', and 'e=0.1 kg'. There are also icons for information (i), Wi-Fi, and battery status in the top right corner.

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This screenshot is identical to the one above, showing the same medical scale interface with patient/nurse fields, PRE-TARE ID 3, weight -13.5 kg, height/BMI fields, and max/min/accuracy specifications at the bottom.

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EXIT General			
Auto Off Time	180s	WiFi	—
Backlight	High	G-Compensation	Off
Buzzer	On		
Data Transfer	On		
Date/Time	—		

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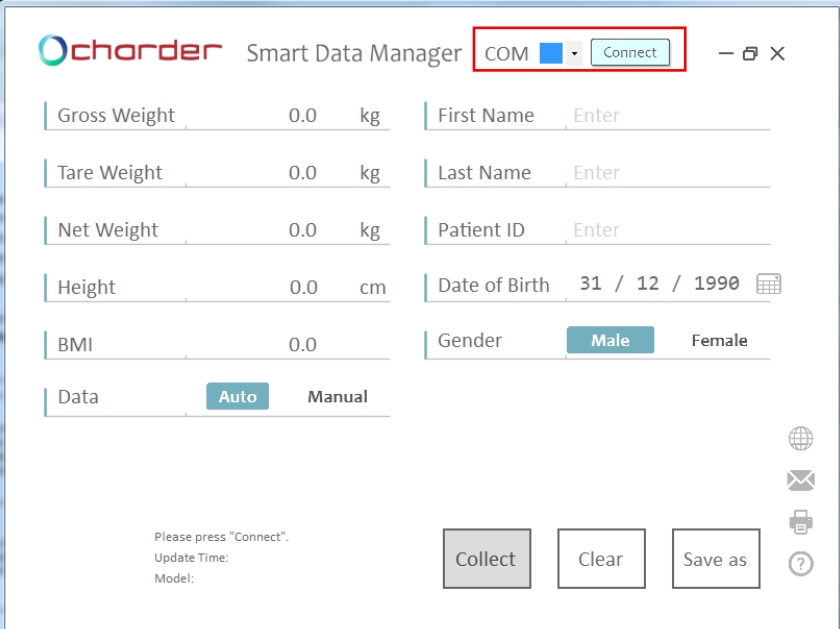
The screenshot displays the Charder Smart Data Manager software interface. At the top, the Charder logo is on the left, followed by the text "Smart Data Manager". To the right of this text is a dropdown menu labeled "COM" with a blue square icon, and a "Connect" button. The "Connect" button is highlighted with a red rectangular box. Below the header, there are several input fields arranged in two columns. The left column includes "Gross Weight" (0.0 kg), "Tare Weight" (0.0 kg), "Net Weight" (0.0 kg), "Height" (0.0 cm), "BMI" (0.0), and a "Data" section with "Auto" and "Manual" buttons. The right column includes "First Name" (Enter), "Last Name" (Enter), "Patient ID" (Enter), "Date of Birth" (31 / 12 / 1990), and "Gender" (Male/Female). At the bottom, there is a status area with the text "Please press 'Connect'. Update Time: Model:" and three buttons: "Collect", "Clear", and "Save as". On the far right, there are icons for a globe, an envelope, a printer, and a help/question mark.

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The screenshot shows the 'chorder Smart Data Manager' interface. On the left, there are input fields for 'Gross Weight' (0.0 kg), 'Tare Weight' (0.0 kg), 'Net Weight' (0.0 kg), 'Height' (167.0 cm), and 'BMI' (0.0). Below these is a 'Data' section with 'Auto' and 'Manual' buttons. On the right, there are fields for 'First Name' (Jane), 'Last Name' (Doe), 'Patient ID' (20190201), 'Date of Birth' (31 / 12 / 1965), and 'Gender' (Male/Female). A red box highlights the 'Height' field and the patient information fields. At the bottom, there are 'Collect', 'Clear', and 'Save as' buttons, along with a status message: 'Please press "Connect". Update Time: Model:'.

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The screenshot shows the 'chorder Smart Data Manager' interface after a measurement. The 'Gross Weight' is now 72.5 kg, 'Net Weight' is 72.5 kg, and 'BMI' is 26.0. The 'Height' remains 167.0 cm. The 'Data' section shows 'Auto' selected. The patient information fields remain the same. A red box highlights the weight and BMI fields. At the bottom, there are 'Collect', 'Clear', and 'Save as' buttons, along with a status message: 'Data updated. Update Time: 06/03/2020 11:40:05 Model:'.

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