

Quick Guide for MS3200




Important : Before operations, make sure you read and understand the manual.

Caution : *Should keep indoor weighing, without water
 Stepping on in the middle part, should not step on the corner
 Should place the scale on the flat surface*




Steps :

1 YOU WILL NEED ...

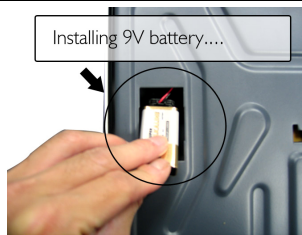

9V *1 battery or

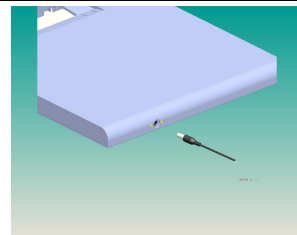

adaptor

2 Put the scale on the flat surface



3 Turn on the scale.
 Press ZERO to zero the scale.



4 Press HOLD before stepping on the scale.
 After beep sound, step down.
 Measurement is completed !!



KEY FUNCTIONS (Model MS3200)

Key	Description
ON/ZERO/OFF	<ul style="list-style-type: none">● Turn ON/OFF the scale● Zero the scale
HOLD	To hold weighing result
UNIT	Unit switch between kilograms, pounds or ounces



Charder Electronic Co., Ltd
103 Guo Zhong Road. Dah Li Dist. Taichung City. Taiwan 41262
TEL:+886 4 24063766
FAX:+886 4 24065612
www.chardermedical.com