



Quick Guide for Indicator DP2700 (model MS3400-1)

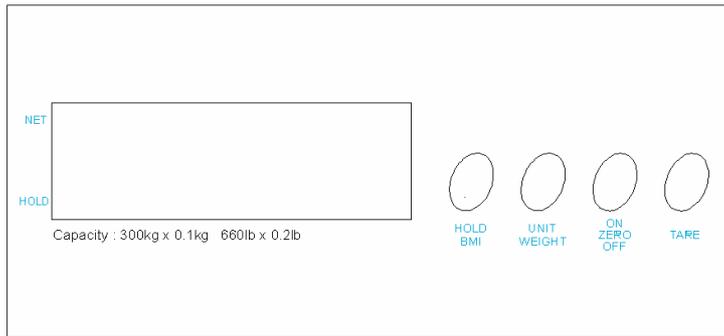

Important : Before operations, make sure you read and understand the manual.


Caution : Should keep indoor weighing, without water
Stepping on in the middle part, should not step on the corner
Should place the scale on the flat surface

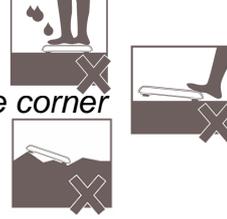

Steps :

1

YOU WILL NEED ...

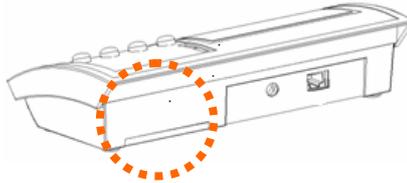

AAA x6 battery

2

Put the scale on the flat surface




3

Turn on the scale.
Press ZERO to zero the scale.



4

Press  before stepping on the scale.
After beep sound, step down.
Measurement is completed !!

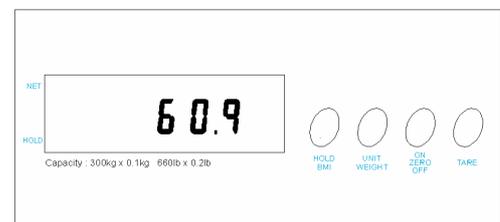

KEY FUNCTIONS (DP2700 DISPLAY- MS3400-1)

Key	Description
HOLD/BMI	<ul style="list-style-type: none">● Determine Body Mass Index● To hold weighing result
UNIT/WEIGHT	<ul style="list-style-type: none">● Weight unit switch between kg and lb
ON/ZERO/OFF	<ul style="list-style-type: none">● Turn ON/OFF the scale● Zero the scale
TARE	Tare un-wanted weight



Charder Electronic Co., Ltd
103 Guo Zhong Road. Dah Li Dist. Taichung City. Taiwan 41262

TEL:+886 4 24063766

FAX:+886 4 24065612

www.chardermedical.com