



Quick Guide for Indicator DP2701 (MS5410)

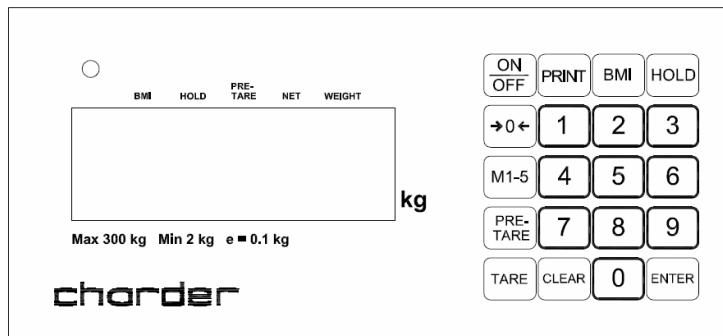

Important : Before operations, make sure you read and understand the manual.


Caution : Should keep indoor weighing, without water
Stepping on in the middle part, should not step on the corner
Should place the scale on the flat surface

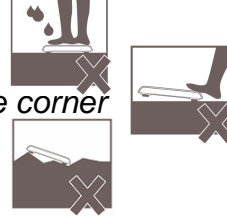

Steps :

1

YOU WILL NEED ...

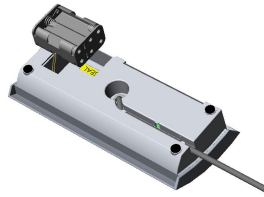

AAA x6 battery

2

Put the scale on the flat surface




and keep bubble level in the center



3

Turn on the scale.

Press  to zero the scale.

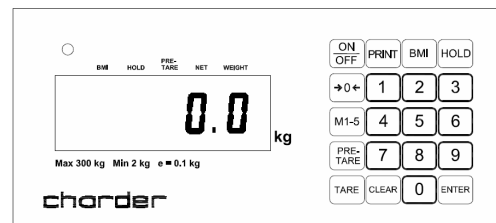


4


Press HOLD before stepping on the scale.

After beep sound, step down.

Measurement is completed !!

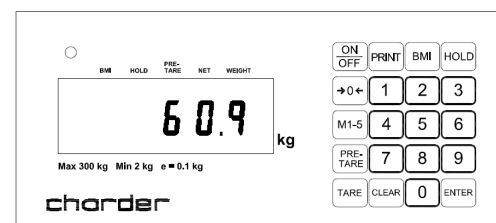

KEY FUNCTIONS (DP2701 DISPLAY- MS5410)

Key	Description
ON/OFF	Turn ON/OFF the scale
PRINT	Print-out the results
BMI	Determine Body Mass Index
HOLD	To hold weighing result
	Zero the scale
M1-5	To store 5 pre-tare values
PRE-TARE	Setup preset tare weight before weighing
TARE	Tare weight during weighing process
CLEAR	To clear the wrong entry
ENTER	To confirm the performing functions.
0-9	Entering digits



Charder Electronic Co., Ltd
103 Guo Zhong Road. Dah Li Dist. Taichung City. Taiwan 41262
TEL:+886 4 24063766
FAX:+886 4 24065612
www.chardermedical.com