


Quick Guide for Indicator DP2701 (MS5710)




Important : Before operations, make sure you read and understand the manual.

Caution : Should keep indoor weighing, without water
Stepping on in the middle part, should not step on the corner
Should place the scale on the flat surface

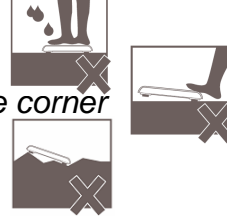


Steps :

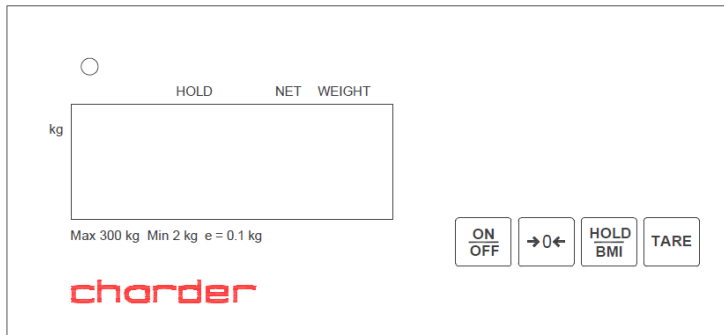

1


YOU WILL NEED ...

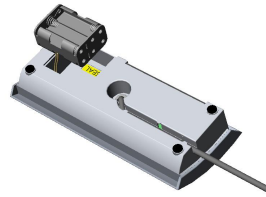

AA x6 battery

2

Put the scale on the flat surface

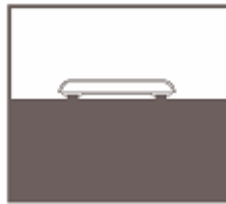



and keep bubble level in the center




3

Turn on the scale.

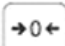Press  to zero the scale.

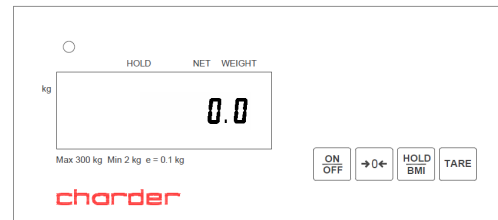


4






Press  before stepping on the scale.

After beep sound, step down.

Measurement is completed !!

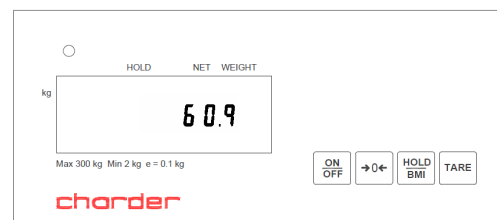

KEY FUNCTIONS (DP2701 DISPLAY- MS5710)

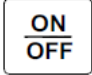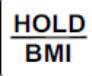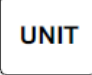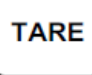Key	Description
	Turn ON/OFF the scale
	<ul style="list-style-type: none">● Determine Body Mass Index● To hold weighing result
	Select measuring units between kg and lb
	Tare un-wanted weight
	Zero the scale



Charder Electronic Co., Ltd
103 Guo Zhong Road. Dah Li Dist. Taichung City. Taiwan 41262
TEL:+886 4 24063766
FAX:+886 4 24065612
www.chardermmedical.com