
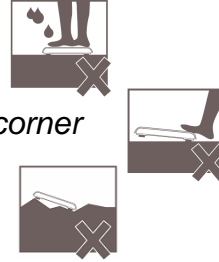


Quick Guide for Indicator DP3700 (MS6000.MS6001)



 Important : Before operations, make sure you read and understand the manual.

Caution : Should keep indoor weighing, without water
Stepping on in the middle part, should not step on the corner
Should place the scale on the flat surface



Steps :


1 YOU WILL NEED ...

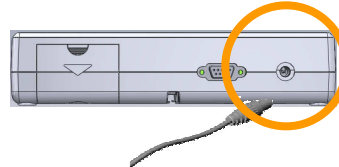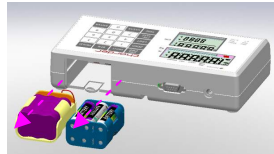



Rechargeable or AA x6 battery **or** adaptor


2 Put the scale on the flat surface
and keep bubble level in the center

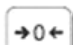

3 Turn on the scale.
Press  to zero the scale.

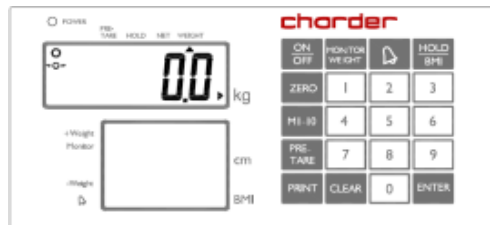

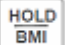4 Press  before stepping on the scale.
After beep sound, step down.
Measurement is completed !!

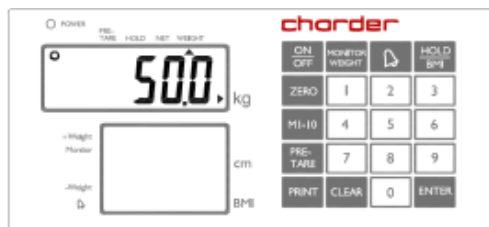

KEY FUNCTIONS (DP3700 DISPLAY- MS6000.MS6001

Key	Description
ON/OFF	Turn ON/OFF the scale
ZERO	Zero the scale
M1-10	To store 10 pre-tare values
PRE-TARE	Setup preset tare weight before weighing (such as bed weight, or any unwanted objects)
HOLD/ BMI	<ul style="list-style-type: none">● HOLD: To hold weighing result● BMI : Determine Body Mass Index
PRINT	Print-out the results
MONITOR WEIGHT	To trace the change in weight of patient
ALARM	To turn ON/OFF and adjust volume of alarm function
CLEAR	To clear the wrong entry
ENTER	To confirm the performing functions.
0-9	Entering digits



Charder Electronic Co., Ltd
103 Guo Zhong Road. Dah Li Dist. Taichung City. Taiwan 41262
TEL:+886 4 24063766
FAX:+886 4 24065612
www.chardermedical.com