

charder®

HM80P PortStad

Height Measurement

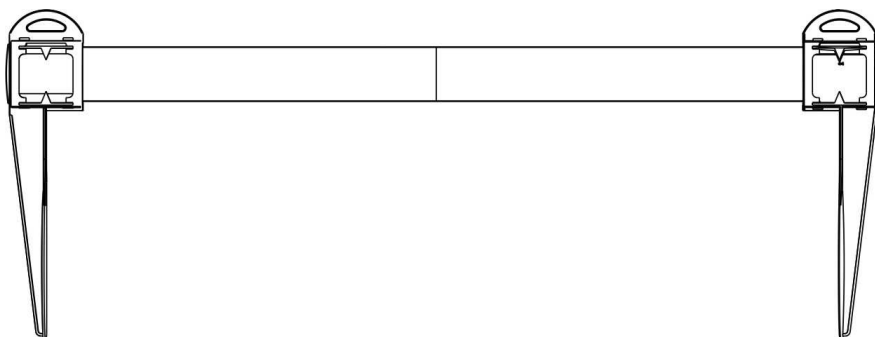

Please keep the instruction manual at hand all the time for future reference.

TABLE OF CONTENTS

PREFACE	2
SAFETY INSTRUCTIONS.....	2
CLEANING.....	2
NOTE BEFORE USE	2
WARRANTY.....	3
EXPLANATION OF THE GRAPHIC SYMBOLS.....	4
SPECIFICATION.....	5
HM80P WALL MOUNTED INSTRUCTION.....	5

PREFACE

Thank you for choosing CHARDER MEDICAL product. All features of this product were designed to state of the art and are optimized for simple and straightforward use. If you have any queries or experience any problems not addressed in the operating instructions, please contact your CHARDER MEDICAL service partner, or visit us on the Internet at www.chardermedical.com

SAFETY INSTRUCTION



Before putting the device into use, please read with care the information given in the Operating Instructions. They contain important instructions for installation, proper use and maintenance of the device.

The manufacturer shall not be liable for damages arising out of failure to heed the following instructions:

- ◆ Improper installation will render the warranty null and void.
- ◆ This device is designed for use indoors.
- ◆ Observe the permissible ambient temperatures for use

If you have any problem, please contact your local CHARDER MEDICAL service partner.

CLEANING

- ◆ We would recommend using alcohol based wipes or similar when cleaning the scales.
- ◆ Please do not use large amounts of water when cleaning the device as this will cause damage to the device.

You should also refrain from using corrosive liquids or high pressure washers.

NOTE BEFORE USE

Before using measuring rod, please take time to read the following safety instructions:

- Follow the operating instructions.
- Use the assembly material supplied.
- When assembling, ensure that the rod has been assembled correctly (See chapter “Assembly instructions”).

WARRANTY

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects, which come under warranty, will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honored only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honored if the equipment is opened by persons not expressly authorized by Charder to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

PLEASE VISIT WWW.CHARDERMEDICAL.COM FOR MORE INFORMATION.

EXPLANATION OF THE GRAPHIC SYMBOLS

SN-T13000001



Designation of the serial number of every device, applied at the device.
(Number as an example)
“Please note the accompanying documents“
or “Observe operating instructions”



Charder Electronic Co., Ltd.
No.103, Guozhong Rd., Dali Dist.,
Taichung City 412, Taiwan (R.O.C.)


Identification of manufacturer of medical product including address



Dispose of old appliances separately from your household waste!!!
Instead, take them to communal collection points.



Carefully read this operation manual before setup and commissioning, even if you are already familiar with Charder scales.

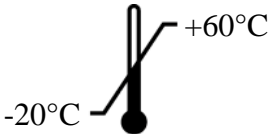


Transport and storage temperature limit indicating the upper and the lower limit (Transport and storage temperature on packaging)

SPECIFICATION

Model	HM 80P
Measuring range	10 ~ 80 cm / 3 $\frac{15}{16}$ ~ 31 $\frac{1}{2}$ inch
Graduations	1 mm / $\frac{1}{16}$ inch
Accuracy	± 10 mm
Dimensions (WxHxL)	89 x 33.5 cm / 35.0" x 13.2"
Weight	Approximate 688g / 1.5 lb
Temperature range	5°C up to 35°C


HM80P WALL MOUNTED INSTRUCTION

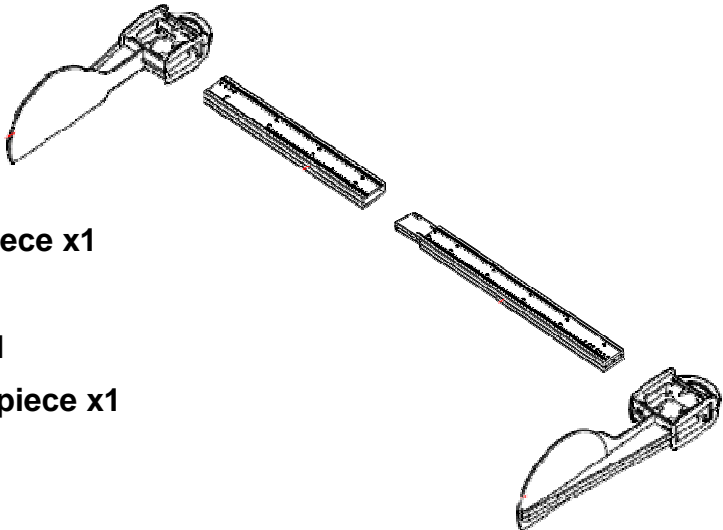

1. Left head piece x1
2. Left rod x1
3. Right rod x1
4. Right head piece x1


One side measurement in centimeters; the other side shows inches.

How to use – for left-handed

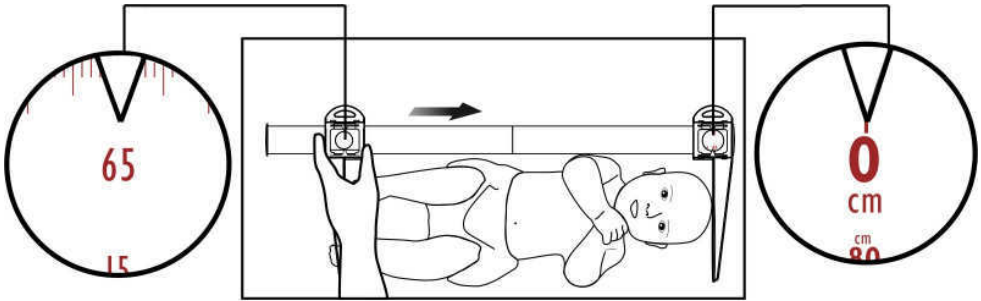


1. Fix right head piece at 0cm.
2. Straighten baby's feet.
3. Slide left head piece till touch the sole of baby.
4. The reading on left head piece is the length of baby.

How to use – for right handed

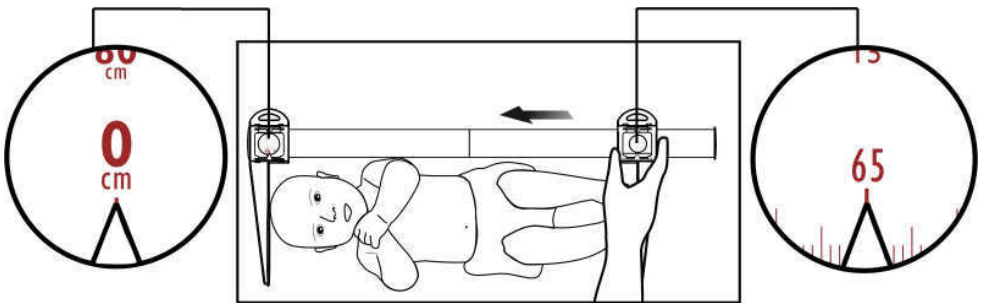


1. Fix left head piece at 0cm.
2. Straighten baby's feet.
3. Slide right head piece till touch the sole of baby.
4. The reading on right head piece is the length of baby.

How to use – for left-handed

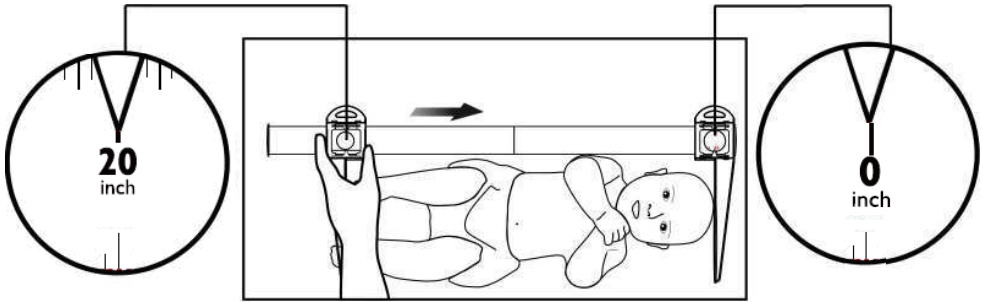

1. Fix right head piece at 0 inch.
2. Straighten baby's feet.
3. Slide left head piece till touch the sole of baby.
4. The reading on left head piece is the length of baby.

How to use – for right handed

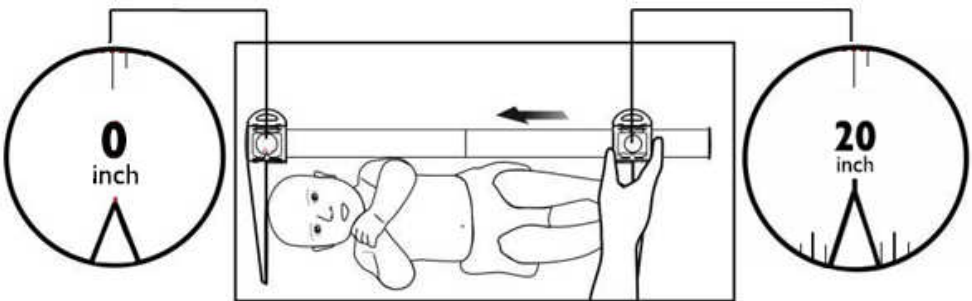


1. Fix left head piece at 0 inch.
2. Straighten baby's feet.
3. Slide right head piece till touch the sole of baby.
4. The reading on right head piece is the length of baby.

NOTE

Manufacturer's Declaration of Conformity

This product has been manufactured in accordance with the harmonized European standards, following the provisions of the below stated directives:

	93/42/EEC as amended by 2007/47/EC Medical Device Directive
--	--

Please see separate document showing on sticker of device for above CE marking.

Authorized EU Representative:



Wellkang Ltd
Suite B, 29 Harley Street
LONDON, W1G 9QR, U.K.

Manufactured by:



Charder Electronic Co., Ltd.
No.103, Guozhong Rd., Dali Dist.,
Taichung City 412, Taiwan (R.O.C.)

FDA no.: D072478